










Instruction card

Reflex

Safety Instructions


- Read the user manual carefully before use.
- Maximum load: 135 kg.
- Ensure that the brakes are applied to the chair during patient transfers to and from the chair.
- Set the leg/foot support to the correct height
- Adjust head support correctly (especially when tilting)
- Armrests closed during height adjustment, tilting and manoeuvring.

1a Operation

Up		
Down		
Backward tilt		
Forward tilt		
Activate	 + 	1 sec
Stop button		
Activate	 + 	4 secs

Important!


This instruction card should not be seen as a replacement for the user manual

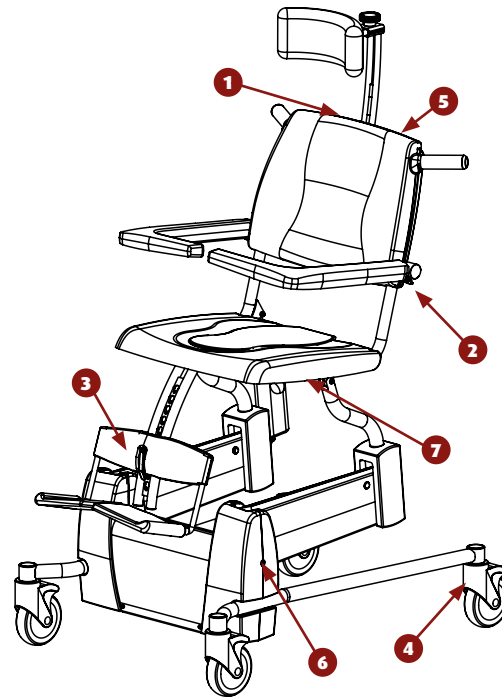

1b Operation


1. Activate the Reflex by holding down the two 'up' buttons at the same time for 1 second
CAUTION: After use, the Reflex will go into stand-by mode for 60 minutes.
To activate see step 1.
2. Set the chair to the correct position, including the head and leg rests (or fold these up).
3. Apply brakes on the chair!
4. Open the armrests and, if necessary, fold up.
5. Support patient when they sit down.
6. Close the armrests.
Getting out of chair occurs in reverse order
In the event of an emergency, all functions on the Reflex can be deactivated by using the stop button.
To reactivate the chair, hold down the two 'up' buttons together for 4 seconds.

2 Armrests

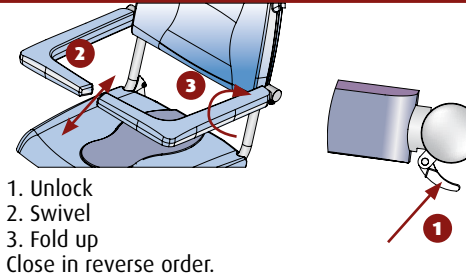


1. Unlock
 2. Swivel
 3. Fold up
- Close in reverse order.

3 Leg support

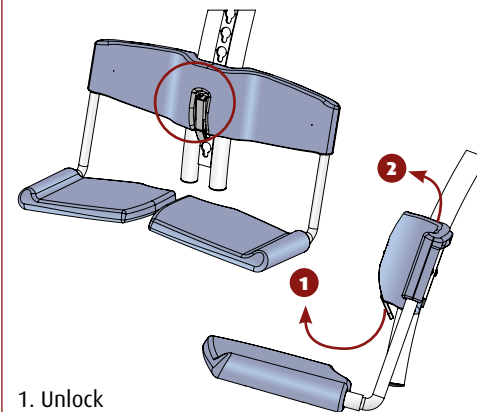

1. Unlock
2. Remove complete leg support
3. Select required position
4. Hook in and lock

4 Brakes

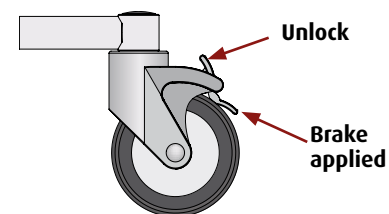

5 Charging

After use, attach chair to the charger for upkeep of the batteries.




Off
Green
Red light flashing
Red light on immediately

Stand-by
Chair operational
Battery almost empty
Battery empty, charge

The chair cannot be operated during recharging.

6 Emergency lowering

If no functions are operating, the chair can be lowered using the key provided.

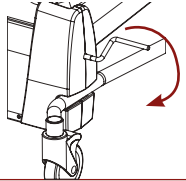

7 Bucket or bedpan holder

When using a bucket or bedpan, the seat cover needs to be removed and the bucket or bedpan inserted from behind, under the seat, in the special holder before the user sits down on the chair.

Cleaning

After use each time, the Reflex must be cleaned with hot water and a normal household detergent. Do not use a scourer or any other aggressive solvent. Check the wheels regularly for any hairs, dirt and remains of soap and remove if necessary.

In the event of defects, contact your dealer.