
Use

1. Apply the brake to the chair!
2. Fold the foot rests up.
3. If necessary, fold the armrests up.
4. Support the less able person when they are taking a seat.
5. Shut the armrests and let the client's feet rest on the foot plates.

Stand up by carrying out the same procedure in reverse order.

Wheel blocking

1. Wheel blocking.
2. Unlocking.

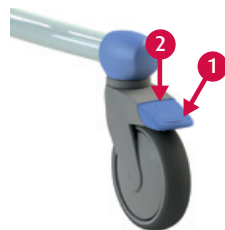Foot rest

Both foot plates must be folded away for the less able person to take a seat in the chair. To do so, fold up the foot plates (1) or turn away both foot supports (2). Make sure that the person places both feet on the foot plates when the chair is moving.

Armrest

1. Fold up armrests.

Toilet function

- Remove the lid on the seat before the person sits down in the chair.
- *In combination with toilet:* The Tango can be moved over almost every toilet.
- *With bedpan:* The Tango can also be used as a toilet chair. Simply slide the stainless steel / plastic bedpan under the chair using the guides.

Cleaning and maintenance

The Tango must be cleaned after every use, with water and regular household cleaning agents. Do not use any abrasive or other aggressive cleaning products. Check the wheels regularly for hairs, dust and soap residues and remove when necessary. The complete maintenance schedule can be found in the instructions for use (**find this at: www.lopital.com**).

In the event of a breakdown, contact Lopital.
Tel: +31 (0)13 - 5239300, **Fax:** +31 (0)13 - 5239301
E-mail: service@lopital.nl

Instructions for use

For the full instructions for use go to **www.lopital.com**

Safety recommendations

- Read the instructions for use carefully (find this at: www.lopital.com) before use.
- Maximum load 180 kg.
- Make sure that the chair is on the brake during client transfers from and to the device.
- Place both feet on the foot supports.

Important!

This instruction card may not be considered a replacement for the instructions for use.

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