

P:\SW-Ftek\Timo en Cleo\Timo 6020\

Signed	KA	Date	01-03-2016
Revised	01 /KA	Date	29-10-2018
Revised	02 / KA	Date	31-03-2020

Overview parts Timo

Copyright reserved
by law


Unit of measure-
ment: mm

Lopital Nederland bv

Type nr: 54006020

Timo 54006020

Part nr.	QTY.	Art.nr.	Part
1	1	70602101	Frame
2	1	70203509	Backrest
3	1	70602007	Seat
4	1	70565001	Armrest
5	4	70602006	Snaploc
6	4		Tube cap Ø25
7	4		Tube cap Ø25
8	2	75005640	Cylinder head screw M8x16 Super Duplex 4529
9	4		Countersunk screw M6x10
10	4		Capring M4
11	4		Chipboard screw 4,2x30
12	4		Screw cap M4

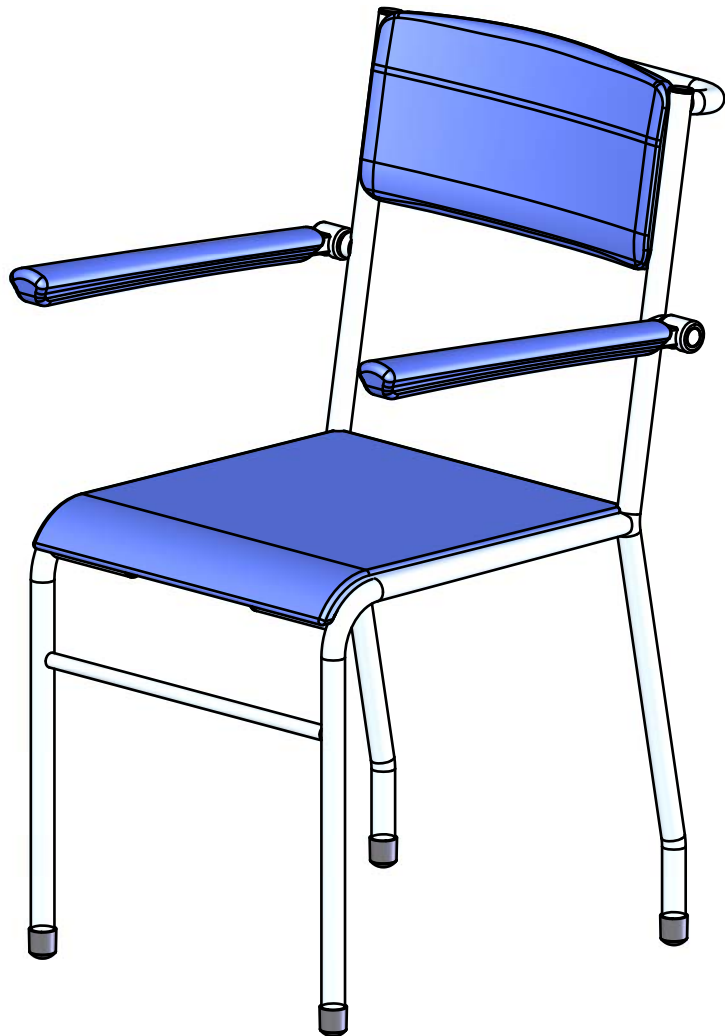

P:\SW-Tek\Timo en Cleo\Timo 6020\

Signed	KA	Date	01-03-2016
Revised	01 /KA	Date	29-10-2018
Revised	02 / KA	Date	31-03-2020

Overview parts Timo

Copyright reserved
by law

Unit of measure-
ment: mm

Lopital Nederland bv

Type nr: 54006020