Creating balance in healthcare

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6100 2400 / 6100 2401  
Sirocco Shower stretcher

**Important:**

Read these instructions carefully before using the shower stretcher / dressing table!

Date first used .....

User .....

Name .....

Address .....

**Manufacturer**

Lopital Nederland B.V.  
 Laarakkerweg 9  
 5061 JR Oisterwijk  
 The Netherlands  
 T +31 (0)13 5239300  
 F +31 (0)13 5239301  
 www.lopital.com  
 info@lopital.nl

All of the aids produced by Lopital fulfil the following norms:  
 MDD 93/42 EEG, NEN-EN-ISO 14971, NEN-EN 12183, ISO 17966, ISO 9001:2015, ISO 13485:2016/  
 C12:2018, NEN-EN 12182, ISO 60601, EN ISO 60601-1-2, EN ISO 60601-1-6, EN ISO 62366, CAN/CSA 60601, ANSI/AAMI ES 60601-1

Notes on symbols	
	Read the user instructions before use.
	Direct current 24V DC.
	Note.
	Applied parts.
	Class 2 EMC.

# Contents

1.	Introduction.....	27
1.1.	Introduction.....	27
1.2.	Safety.....	28
1.3.	Warranty.....	29
2.	Use.....	30
2.1.	First use.....	30
2.2.	Activating.....	30
2.3.	Working with the shower stretcher / dressing table.....	32
3.	Operation.....	32
3.1.	Side bar.....	32
3.2.	Moving upwards / downwards.....	33
3.3.	Water drainage.....	33
3.4.	Changing battery.....	33
3.5.	Charging battery.....	34
4.	Safety functions.....	35
4.1.	Emergency stop button.....	35
4.2.	Sleep setting.....	35
4.2.	Emergency lowering button.....	35
5.	Cleaning and maintenance.....	36
6.	Storage and transport.....	37
7.	Removal of parts.....	37
8.	Technical specifications.....	37
8.1.	Structure.....	37
8.2.	Drive.....	38
8.3.	Dimensions.....	38
9.	Checklist for technical problems.....	40
10.	Relationship with other supports.....	41
11.	Electromagnetic compatibility.....	42

# 1. Introduction

## 1.1. Introduction

The shower stretcher / dressing table is a medical device for caring for and washing clients in an ergonomically responsible manner. Comfort is the primary consideration for both the client and the carer.

Lopital supplies various versions of its shower stretchers and dressing tables. The range includes stretchers / dressing tables that are secured to a wall, mobile shower stretchers / dressing tables and shower stretchers / dressing tables that are non-mobile. The height of some variants is electrically or hydraulically adjustable.

The shower stretcher / dressing table is multi-functional in use and corresponds to the use of other devices such as beds, wheelchairs and shower chairs, hoisting equipment and sliders such as the EasySlide. These are the essential functions of a shower stretcher / dressing table.

The average technical life of the shower stretcher / dressing table is 10 years, assuming it is cleaned daily and regularly maintained according to the maintenance schedule supplied (*See chapter 5 Cleaning and maintenance*).

*The shower stretcher / dressing table is a class 1 product with CE-marking. This marking corresponds with directive 93/42/EC and the revised directive 2007/47/EC for medical devices of 21 March 2010. The shower stretcher / dressing table has been tested for electromagnetic radiation according to norm 60601-1-2.*

The Amfora is available in the following versions:

- 6200 5005 – Electric high / low, lying surface 160 cm / 63 inch.
- 6200 5010 – Electric high / low, lying surface 190 cm / 74.8 inch.

The Vienna is available in the following versions:

- 6500 5015 – Electric high / low, lying surface 160 cm / 63 inch.
- 6500 5020 – Electric high / low, lying surface 190 cm / 74.8 inch.

The Sirocco is available in the following versions:

- 6100 2400 – Wall mounted left and height adjustable
- 6100 2401 – Wall mounted right and height adjustable

## 1.2. Safety

**A failure to comply with the following safety points and the further definitions in this user manual could lead to hazardous situations. Please read carefully before use.**

### *Use*

*The following applies to all shower stretchers and dressing tables:*

- It is not permitted to load the shower stretcher / dressing table with more weight than 150 kg / 330 lbs (*The maximum load depends on the strength of the wall*).
- Only use original parts, supplied by Lopital.
- Use the shower stretcher / dressing table in a clean and tidy working environment.
- Only verifiably authorised personnel is allowed to work with the shower stretcher / dressing table.
- Swimming pool locations are not considered to be 'normal conditions' and will substantially shorten the lifespan of certain components.
- The shower stretcher / dressing table must be used / cleaned using water that is below 43°C (109°F).
- The shower stretcher / dressing table may only be used indoors.
- Notify the client before moving the shower stretcher / dressing table.
- The shower stretcher / dressing table may not be used to transport loads other than people.
- Be aware of obstacles when moving upwards and downwards.
- The use of components (ladders, cables, actuators, battery, etc.) that have not been supplied by Lopital may result in increased emissions or decreased immunity of the shower stretcher / dressing table.
- Charge battery in a well-ventilated, dry space.
- Portable and mobile RF communication equipment can affect medical devices.
- Medical electrical devices involve special EMC precautions and must be used as per the EMC information outlined in the instructions for use.
- The shower stretcher / dressing table must not be used adjacent to or stacked on other products. However, should that be necessitated, it must be checked whether the stretcher functions normally in this configuration too.

### Maintenance

- If the charger or cable become damaged, they must be replaced.
- The shower stretcher / dressing table must be inspected and serviced at least once a year.
- Servicing and maintenance of the shower stretcher / dressing table may only be carried out by persons authorised by Lopital.
- Constructive components of the shower stretcher / dressing table may only be replaced by suitably authorised Lopital personnel.
- If erroneous use, transport, an accident or improper maintenance leads to deformation of the shower stretcher / dressing table, it may no longer be used and you must contact the supplier.

In the event of breakdowns, you must contact Lopital or your local supplier.

E-mail: [service@lopital.nl](mailto:service@lopital.nl)

**The shower stretcher / dressing table must only be operated by persons who have been sufficiently trained and expert personnel who have experience working with the target group.**

**The shower stretcher / dressing table must be used in a clinical setting.**

**Making changes to the construction could affect the safety of the shower stretcher / dressing table. This will also lead to Lopital's liability and warranty conditions being invalidated. As a result, the shower stretcher / dressing table will no longer comply with the guidelines for medical devices.**

### 1.3. Warranty

Lopital products are guaranteed for 2 years with respect to construction and / or material faults. In the event of varying warranty terms, the exact period will be indicated on your invoice; this also serves as the warranty itself.

Components that are subject to wear and tear (such as battery, hand controls and cables) will not fall under warranty unless there are clear construction and / or material faults.


## 2. Use

### 2.1. First use

Before you use the shower stretcher / dressing table, you must check to ensure the following components are present. These components can be replaced by authorised individuals in the event of damage. These components are the applied parts of the shower stretcher / dressing table. These components may come into contact with the user(s) while the shower stretcher / dressing table is in use.

### 2.2. Activating

The shower stretcher / dressing table must be activated before use, which can be done by pressing any button on the hand control for 2 seconds.

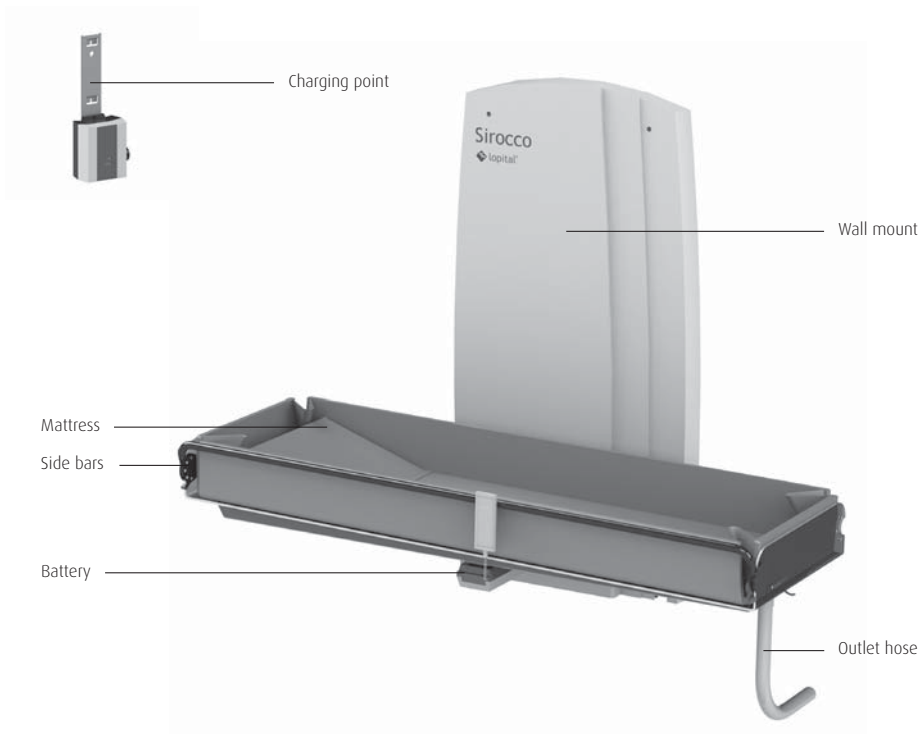## 2.3. Working with the shower stretcher / dressing table

### Step 1

Put the shower stretcher / dressing table in the correct position in order to ease transferring the client (See paragraph 3.2. Moving upwards / downwards).

### Step 2

Lower the side rail of the shower stretcher / dressing table (See paragraph 3.1. Side bars).

### Step 3

The client can be moved onto the shower stretcher / dressing table.

### Step 4

Close the side bar.

## 3. Operation

### 3.1. Side bar

- The shower stretcher / dressing table is equipped with steel, hinged side bars.
- The side bar is equipped with safeguards so that the less able person cannot raise the side bars himself.
- Before the side bar is collapsed downwards, the safeguard must be unlocked.
- Unlock the safeguard by removing the bolts on both sides of the side bar (1).
- Turn the unlocked side bar outwards (3).
- Move the side bar upwards in order to lock it in place again. The side bar is locked on the mobile shower stretcher / dressing table by securing the bolts in the appropriate position.

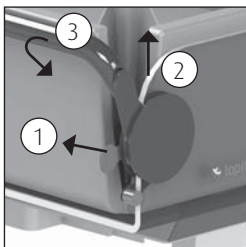
Figure 1 Side bar Sirocco.

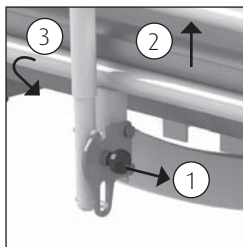
Figure 2 Side bar Amfora / Vienna wall.

## 3.2. Moving upwards / downwards

### *Moving upwards*

- Move the shower stretcher / dressing table upwards by pressing the button that shows an 'up' arrow. The shower stretcher will then immediately move upwards (See figure 3).
- You can stop the movement whenever you like by taking your finger off the button.
- During the upward movement, maintain contact with the client.
- If the manual control is not being used, it can be hung on the side bar.
- Ensure that the curly cable on the operating device does not get jammed.

Figure 3 Moving upwards.

### *Moving downwards*

- Move the shower stretcher / dressing table downwards by pressing the button that shows a 'down' arrow. The shower stretcher will then immediately move downwards (See figure 4).
- You can stop the movement whenever you like by taking your finger off the button.
- During the downward movement, maintain contact with the less able person.

Figure 4 Moving downwards.

**Before moving the shower stretcher / dressing table upwards, check to ensure there are no persons or objects in the vicinity of the device.**

## 3.3. Water drainage

- Water is drained away from the shower stretcher via a drainage hose.
- Take the hose off the hook and direct it to a drainage point that is as low as possible.
- During transport, always hang the hose on the hook otherwise it may leak and drip.

## 3.4. Changing battery

- If you hear an audible signal, the battery must be changed.
- The battery is located under the.
- In order to remove the battery, use one hand to unlock the catch (1) on the battery, then slide the battery to the right (2) (See figure 5).
- To install the fully charged battery, it must first be positioned next to the electronic housing and then clicked into the holder.

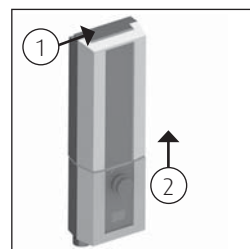
Figure 5 Battery.

**Make sure that the shower stretcher / dressing table is always ready for use by having a fully charged reserve battery on standby.**

### **3.5. Charging battery**

- Place the empty battery on the battery charger (See figure 6).
- A green light indicates that the battery charger is switched on. An orange light indicates that the battery must be charged. This light goes off if the battery is full.
- The charged battery can remain on the battery charger without being damaged.
- In order to remove the battery, use one hand to unlock the catch on the battery and the other to support the battery itself.

Figure 6 Charger.

**Only the original charger for the shower stretcher / dressing table may be used. If you use another charger, Lopital cannot be held liable for any consequences, such as damage to the electronics.**

**The charger that must be used for the shower stretcher / dressing table is the 'Linak External charger CHJ2'.**

**Charge batteries in a well-ventilated, dry space.**

**After using the shower stretcher / dressing table, connect it to the charger to maintain the battery. Charge the stretcher at least once a week!**

For instruction videos go to [www.lopital.com](http://www.lopital.com) or [www.youtube.com/user/LopitalNL](http://www.youtube.com/user/LopitalNL)

## 4. Safety functions

### 4.1. Emergency stop button

At the controlbox, there is an emergency stop button. After the emergency stop has been pressed, all electrical operation will cease to function. Turn the emergency stop a quarter turn to unlock it again (See figure 7).

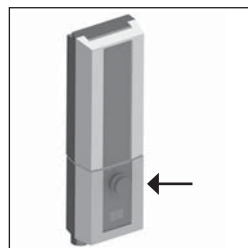
Figure 7 Emergency stop button.

**The orange LED light on the hand control will flash when the Lock button has been activated.**

### 4.2. Sleep setting

After around an hour, the stretcher will fall into a 'sleep' setting and the battery indication LEDs on the manual control will go out. In order to use the stretcher again, it must be activated (See paragraph 2.2. Activating).

### 4.2. Emergency lowering button

If the controls stop working and the shower stretcher / dressing table is in a raised position, it can be lowered using the emergency lowering button. This button is located to the left of the emergency stop button (See figure 8).

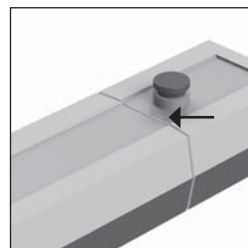
Figure 8 Emergency lowering button.

First check to ensure that the emergency button has not been pressed!

- Remove the plug from the manual control.
- Press the emergency lower button (to the left of the emergency stop).

The shower stretcher / dressing table will now return to the lowest position (make sure no persons or objects are under the stretcher).

Take the less able person from the shower stretcher / dressing table and make sure the device is not used until it has been repaired. Place the shower stretcher / dressing table in a safe place and indicate that it is out of order.

Contact the technical service or Lopital's service department.

E-mail: [service@lopital.nl](mailto:service@lopital.nl)

## 5. Cleaning and maintenance

The shower stretcher / dressing table must be cleaned after every use, with water and regular household cleaning agents. Do not use any abrasive or other aggressive cleaning products. If necessary, the lying surface can be disinfected with a solution of 70% ethanol or propanol.

Maintenance and repairs to the shower stretcher / dressing table may only be carried out by authorised personnel.

Maintenance schedule	
<b>After every use</b>	<ul style="list-style-type: none"> <li>• The shower stretcher / dressing table can be cleaned with water and regular household cleaning products.</li> <li>• Clean and disinfect mattress / harness.</li> </ul>
<b>Weekly</b>	<ul style="list-style-type: none"> <li>• Clean frame.</li> <li>• Check that the shower stretcher / dressing table is still working correctly.</li> </ul>
<b>Annually</b> 	<ul style="list-style-type: none"> <li>• According to the guidelines for medical devices, a shower stretcher / dressing table must be maintained in line with the manufacturer's instructions. This process will preferably be carried out by Lopital or another authorised body. Lopital offers the option of having the shower stretcher / dressing table thoroughly inspected and serviced once a year via Periodic Preventative Maintenance.</li> </ul> <p>For more information about the maintenance contract, please contact Lopital (<a href="mailto:service@lopital.nl">service@lopital.nl</a>) or speak to your supplier.</p>

## 6. Storage and transport

During transport and storage, the ambient conditions must remain moderate. In other words, the temperature should remain between -40°C and +55°C (-40°F to 130°F) and humidity levels should be kept as low as possible. Vibration during transport must be kept to a minimum in order to avoid damaging the hinged mechanisms on the stretcher.

## 7. Removal of parts

All parts that are replaced or removed can be sent back to Lopital BV. You may also offer the parts to the engineer for destruction. We will ensure that all parts are processed in an environmentally-friendly manner.

If you take care of disposal of the product yourself, you must take account of the fact that batteries and chargers must be disposed of as small chemical waste.

## 8. Technical specifications

**This shower stretcher / dressing table belongs to category 'BF'.**

**This shower stretcher / dressing table is a class 2 EMC product**

The shower stretcher / dressing table is splash-proof; this means that it can be used under the shower. This is indicated with instruction IPX4.

### 8.1. Structure

Frame	steel + two layer powder coating
Side bars	steel + two layer powder coating
Hinges	stainless steel + two layer powder coating

## Weight

### Amfora

6200 5005/10      85 / 88 kg      187.4 / 194.0 lbs

### Vienna

6500 5015/20      75 / 78 kg      165.3 / 172.0 lbs

### Sirocco

6100 2400/01      80 kg      176.4 lbs

## Maximum load

150 kg / 330 lbs (The maximum load depends on the strength of the wall)

## 8.2. Drive

High / low drive	Actuator 24V DC
Control box	24VDC, max. 250W
Battery	24V, max. 250W
Charger	230V AC, connection value 100W, wall-mounted

## 8.3. Dimensions

### External dimensions

#### Amfora (l x w x h)

6200 5005/10    160 / 190 x 94 x 49 to 109 cm    63 / 74.8 x 37.0 x 19.3 to 42.9 inch

#### Vienna (l x w x h)

6500 5015/20    160 / 190 x 91 x 49 to 109 cm    63 / 74.8 x 35.8 x 19.3 to 42.9 inch




#### Sirocco (l x w x h)

6100 2400/01    200 x 81 x 43 to 103 cm            78.7 x 31.9 x 16.9 to 40.6 inch

Water drainage hose    Internal Ø 4 cm / 1.6 inch

Wheel base              120 cm / 47.2 inch

Track width              65 cm / 25.6 inch

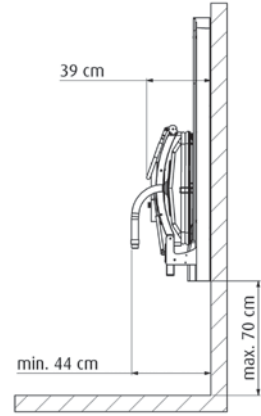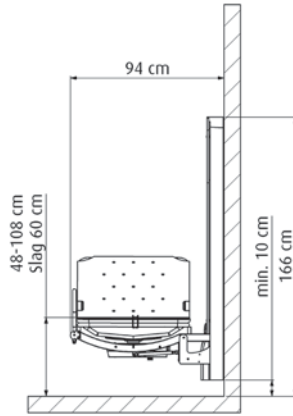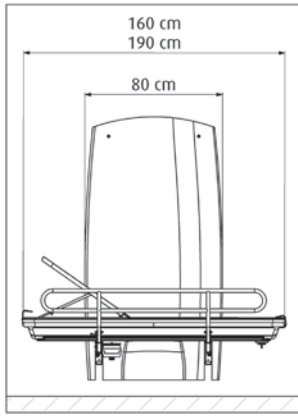## 9. Checklist for technical problems

Problem	Cause	Solution
<b>Shower stretcher / dressing table does not move upwards or downwards</b>	Emergency stop button used.	Turn emergency button 90 degrees to the right.
	Battery empty.	Charge battery / install reserve battery.
	Plugs removed.	Check plugs (to side of control box).
	Manual controls not working.	Check plugs. Contact Lopital Nederland B.V.
<b>Light on the charger does not come on</b>	Charger is not plugged in.	Connect charger.
	Charger is not working.	Contact Lopital Nederland B.V.
<b>Shower stretcher dressing table stops</b>	Emergency stop button used.	Turn emergency button 90 degrees to the right.
	Technical issue.	Contact Lopital Nederland B.V.

For all other issues please contact Lopital Nederland B.V.

Telephone: +31 (0)13 - 5239300

Fax: +31 (0)13 - 5239301

E-mail: [service@lopital.nl](mailto:service@lopital.nl)

## 10. Relationship with other supports

### *EasySlide*

The EasySlide is the ideal device for transferring from bed to shower stretcher / dressing table and vice versa. The physical burden and time required when using the EasySlide is minimal. The EasySlide is suitable for most bed and chair transfers from and to the shower stretcher / dressing table.

### *Hoist*

The adjustable shower stretchers / changing tables are between 50 cm / 19.7 inches in height (hydraulic) and 55 cm / 21.7 inches (electrical) with an underneath docking of 6 cm / 2.4 inches. The non-adjustable shower stretchers / changing tables are 90 cm / 35.4 inch in height. The underneath docking of the fixed stretcher / changing table is 25 cm / 9.8 inches.

### *Wheelchair*

In lower settings of 48-55 cm / 18.9-21.7 inch, patients can be transferred from the adjustable shower stretcher / dressing table to wheelchair and vice versa. An ideal device to be used during this operation is the small EasySlide (and possibly a sliding board).

### *Bed*

The shower stretcher / dressing table is primarily used to wash users while they are lying down. Users can be moved, using the EasySlide, from any (high or low) bed to the shower stretcher / dressing table in a lying position. The shower stretcher / dressing table must be brought to the same height as the bed to carry out this process.

# 11. Electromagnetic compatibility

(For electric variants)

<b>Guidance and Manufacturer's declaration-electromagnetic emissions</b>		
The shower stretcher / dressing table is intended for use in the electromagnetic environment specified below. The customer or the user of the shower stretcher / dressing table should assure that it is used in the following environment:		
<b>Emissions Test</b>	<b>Compliance</b>	<b>Electromagnetic Environment - guidance</b>
RF emissions CISPR11	Group 1	The shower stretcher / dressing table uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR11	Class B	The shower stretcher / dressing table is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Class A	
Voltage Fluctuations/ Flicker emissions IEC 61000-3-3	Complies	

<b>Guidance and Manufacturer's declaration-electromagnetic immunity</b>			
The shower stretcher / dressing table is intended for use in the electromagnetic environment specified below. The customer or the user of the shower stretcher / dressing table should assure that it is used in the following environment:			
<b>Immunity Test</b>	<b>IEC 60601-1-2 Test Level</b>	<b>Compliance Level</b>	<b>Electromagnetic Environment - guidance</b>
Electrostatic discharge (ESD) IEC 61000-4-2	±6 kV contact ±8 kV air	±6 kV contact ±8 kV air	Floors should be made of wood, concrete, or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.
Electrical fast transient/burst IEC 61000-4-4	±2 kV for power supply lines	±2 kV for power supply lines	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	±1 kV line(s) to line(s) ±2 kV line to earth	±1 kV line(s) to line(s) ±2 kV line to earth	Mains power quality should be that of a typical commercial or hospital environment.
Voltage dips, short interruptions, and voltage variations on power supply input lines IEC 61000-4-11	<5% Un (>95% drop in Un) for 0,5 cycle  40% Un (60% drop in Un) for 5 cycles  70% Un (30% drop in Un) for 25 cycles  <5% Un (>95% drop in Un) for 5 seconds	<5% Un (>95% drop in Un) for 0.5 cycle  40% Un (60% drop in Un) for 5 cycles  70% Un (30% drop in Un) for 25 cycles  <5% Un (>95% drop in Un) for 5 seconds	Mains power quality should be that of a typical commercial or hospital environment. The shower stretcher / dressing table is equipped with a battery.
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	3 A/m	3 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.
<b>NOTE</b> Un is the a.c. mains voltage prior to application of the test level			

Guidance and Manufacturer's declaration-electromagnetic immunity			
The shower stretcher / dressing table is intended for use in the electromagnetic environment specified below. The customer or the user of the shower stretcher / dressing table should assure that it is used in the following environment:			
Immunity Test	IEC 60601-1-2 Test Level	Compliance Level	Electromagnetic Environment - guidance
RF Common mode/ Conducted Susceptibility IEC 61000-4-6  Radiated RF Electromagnetic Field IEC 61000-4-3	3 Vrms 150 kHz to 80 MHz  3 V/m 80 MHz to 2,5 GHz	3 V  3 V/m	Portable and mobile RF communications equipment should be used no closer to any part of the shower stretcher / dressing table, including cables than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Recommended separation distance  $d = 1,2 \sqrt{P}$ $d = 1,2 \sqrt{P}$ 80 MHz to 800 MHz $d = 2,4 \sqrt{P}$ 800 MHz to 2,5 GHz  Where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in meters (m).  Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey, <sup>a</sup> should be less than the compliance level in each frequency range. <sup>b</sup>  Interference may occur in the vicinity of equipment marked with the following symbol:  
<b>NOTE 1</b> At 80 MHz and 800 MHz, the higher frequency range applies. <b>NOTE 2</b> These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.			
<b>a</b> Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the shower stretcher / dressing table is used exceeds the applicable RF compliance level above, the shower stretcher / dressing table should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the shower stretcher / dressing table. <b>b</b> Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V/m.			

Recommended separation distance between portable and mobile RF communications equipment and the shower stretcher / dressing table			
The shower stretcher / dressing table is intended for use in the electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the shower stretcher / dressing table can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the shower stretcher / dressing table as recommended below, according to the maximum output power of the communications equipment.			
Rated maximum output power of transmitter  W	Separation distance according to frequency of transmitter  m		
	150 kHz to 80 MHz  $d = 1,2 \sqrt{P}$	80 MHz to 800 MHz  $d = 1,2 \sqrt{P}$	800 MHz to 2,5 GHz  $d = 2,4 \sqrt{P}$
0,01	0,12	0,12	0,24
0,1	0,38	0,38	0,76
1	1,2	1,2	2,4
10	3,8	3,8	7,6
100	12	12	24
For transmitters rated at a maximum output power not listed above, the recommended separation distance $d$ in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where $P$ is the maximum output power rating of the transmitter in watt (W) according to the transmitter manufacturer.			
<b>NOTE 1</b> At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.			
<b>NOTE 2</b> These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.			

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Guidance and Manufacturer's declaration-electromagnetic immunity			
The shower stretcher / dressing table is intended for use in the electromagnetic environment specified below. The customer or the user of the shower stretcher / dressing table should assure that it is used in the following environment:			
Immunity Test	IEC 60601-1-2 Test Level	Compliance Level	Electromagnetic Environment - guidance
RF Common mode/ Conducted Susceptibility IEC 61000-4-6  Radiated RF Electromag- netic Field IEC 61000-4-3	3 Vrms 150 kHz to 80 MHz  3 V/m 80 MHz to 2,5 GHz	3 V  3 V/m	Portable and mobile RF communications equipment should be used no closer to any part of the shower stretcher / dressing table, including cables than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Recommended separation distance  $d = 1,2 \sqrt{P}$ $d = 1,2 \sqrt{P}$ 80 MHz to 800 MHz $d = 2,4 \sqrt{P}$ 800 MHz to 2,5 GHz  Where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in meters (m).  Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey, <sup>a</sup> should be less than the compliance level in each frequency range. <sup>b</sup>  Interference may occur in the vicinity of equipment marked with the following symbol:  
<b>NOTE 1</b> At 80 MHz and 800 MHz, the higher frequency range applies. <b>NOTE 2</b> These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.			
<b>a</b> Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the shower stretcher / dressing table is used exceeds the applicable RF compliance level above, the shower stretcher / dressing table should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the shower stretcher / dressing table. <b>b</b> Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V/m.			

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Guidance and Manufacturer's declaration-electromagnetic emissions		
The shower stretcher / dressing table is intended for use in the electromagnetic environment specified below. The customer or the user of the shower stretcher / dressing table should assure that it is used in the following environment:		
Emissions Test	Compliance	Electromagnetic Environment - guidance
RF emissions CISPR11	Group 1	The shower stretcher / dressing table uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR11	Class B	The shower stretcher / dressing table is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Class A	
Voltage Fluctuations/ Flicker emissions IEC 61000-3-3	Complies	

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**For more information, visit our website: [www.lopital.com](http://www.lopital.com)**

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Lopital Nederland B.V. Laarakkerweg 9, 5061 JR Oisterwijk, Postbus 56, 5060 AB Oisterwijk