### Safety recommendations

- Read the Instructions for use carefully (find this at: [www.lopital.com](http://www.lopital.com)) before use.
- Maximum load 180 kg.
- Make sure that the Marina is on the brake during client transfers from and to the stretcher.

### Electric manual control

Up

Down


Activate

2 seconds

### Hydraulic control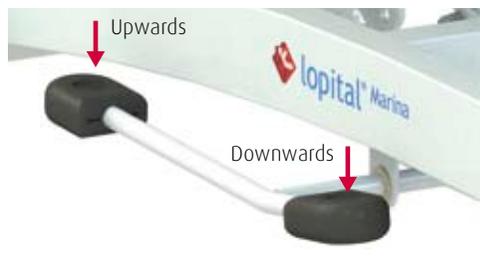

### Important!

This instruction card may not be considered a replacement for the Instructions for use.



### Use


1. Activate the Marina by pressing the 'up' button for 2 seconds.  
**Note:** After use, the Marina will go into a sleep setting after 60 minutes (Electric Marina).
2. Put the stretcher in the right position and at the right height.
3. Make sure the brakes are applied to the stretcher!
4. Fold the side-bars down.
5. Use the EasySlide to slide the person from or to the Marina.
6. Raise the bed base a few centimetres and close the side-bars.
7. Disengage the brake on the Marina.

### Wheel blocking

1. Wheel blocking.
2. Unlocking.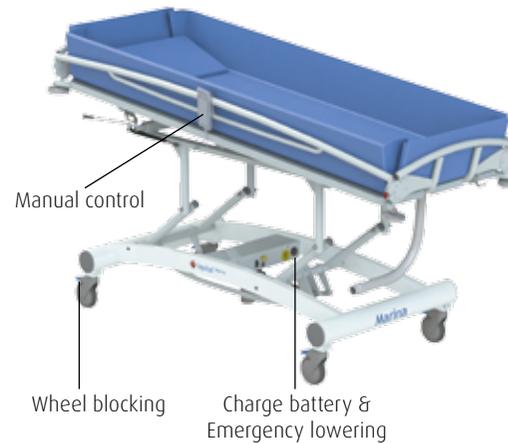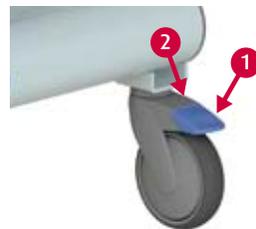

### Side-bars

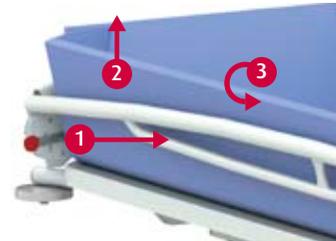1. Unlock.
  2. Tilt.
  3. Turn.
- When closing, the bar locks automatically.

**Make sure that the curly cord from the control panel does not become jammed when closing the side-bar (electric Marina).**

### Charging batteries (Electric Marina)

After using the Marina, connect it to the charger to maintain the batteries.

Off	Sleep setting.
Green	Chair ready to use.
Blinking orange	Emergency stop activated.
Orange	Battery is half empty.
Blinking red	Battery almost empty.
Red	Battery empty.

**The Marina will not work during charging.**

### Emergency lowering (Electric Marina)

If the Marina breaks down, you can lower the stretcher using the emergency lowering button.

### Cleaning and maintenance

The Marina must be cleaned after every use, with water and regular household cleaning agents. Do not use any abrasive or other aggressive cleaning products. Check the wheels regularly for hairs, dust and soap residues and remove when necessary.

The complete maintenance schedule can be found in the Instructions for use (**find this at: [www.lopital.com](http://www.lopital.com)**).

In the event of a breakdown, contact Lopital.  
**Tel:** +31 (0)13 - 5239300, **Fax:** +31 (0)13 - 5239301  
**E-mail:** [service@lopital.nl](mailto:service@lopital.nl)

### Instructions for use

For the full Instructions for use go to **[www.lopital.com](http://www.lopital.com)**

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