

Sirocco Basic & Sirocco Deluxe shower trolley


Instruction card

01. Safety recommendations

- Read the instruction for use before use (www.lopital.com).
- Maximum load 227 kg / 500 lbs (Deluxe) and 150 kg / 330.7 lbs (Basic)




02. Manual control

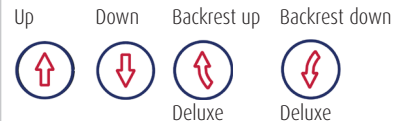

03. Water drainage

Water is drained through the drainage hose. To do this, remove the hose from the recess (1) and route it to the lowest possible drainage point.

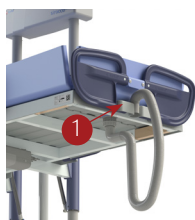

04. Folding up lying surface

The lying surface can easily be folded up to save space.




6500 2510
Sirocco Deluxe



6500 2500
Sirocco Basic




6500 2505
Sirocco Basic
fixed height

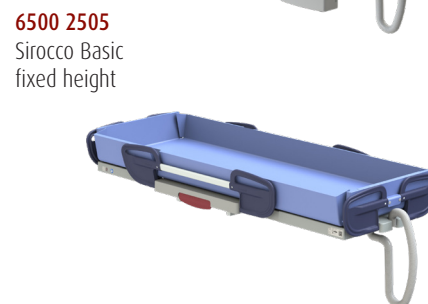

05. Side rail

1. Unlock (pull red handle (1))
2. Rotate the side rail to the desired position: closed / wide / open (2).

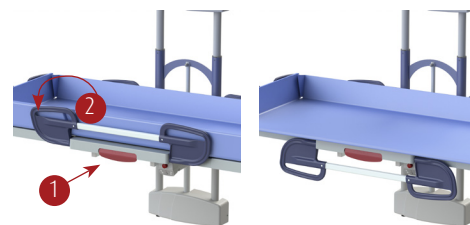

06. Charging battery

Charging the battery before it is completely empty will extend the lifespan.

Battery indicator is located on the hand control.



fully charged



1/3 capacity, charging advised



2/3 capacity



charging required,
an acoustic signal sounds

07. Cleaning and maintenance

The Sirocco Basic / Sirocco Deluxe must be cleaned after every use, with water and regular household cleaning agents. Do not use any abrasive or other aggressive cleaning products. If necessary, the PUR components can be disinfected with a solution of 70% ethanol or propanol. The complete maintenance schedule and cleaning protocol can be found in the instructions for use (www.lopital.com).

According to the regulation MDR (EU) 2017/745 for medical devices, Sirocco Basic / Sirocco Deluxe must be maintained in line with the manufacturer's instructions. Those instructions can be found in the maintenance protocol, see website Lopital (www.lopital.com).

Important!

This quick instruction card may not be considered a replacement for the instruction for use!

In the event of a breakdown, contact Lopital.

Tel: +31 (0) 13-5239300

E-mail: info@lopital.nl

Instruction for use
For the full instruction for
use go to: www.lopital.com

