

**Safety recommendations**

- Read the instructions for use. ([www.lopital.com](http://www.lopital.com)).
- Maximum load 227 kg / 500 lbs.

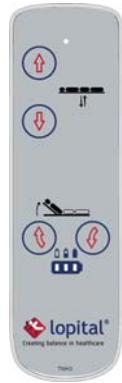

**Manual control**

Up

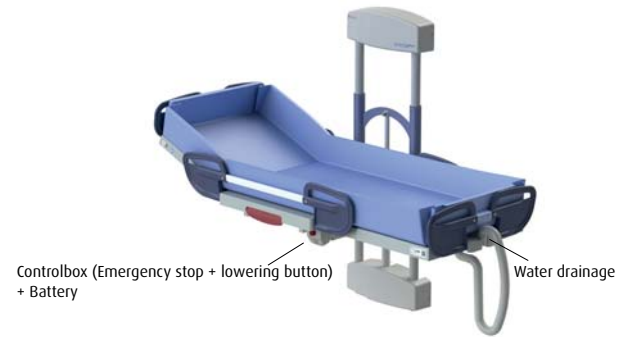
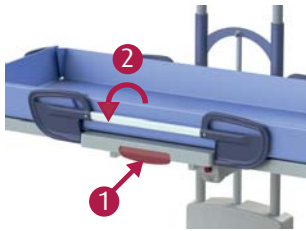


Down


 Backrest up  
(deluxe)

 Backrest down  
(deluxe)

**Side rail**

1. Unlock (Pull red handle)
2. Rotate the side rail to the right position.


**Water drainage**

Water is drained away from the Sirocco basic / Sirocco deluxe via a drainage hose. To do this, remove the hose from the recess (1) and route it to the lowest possible drainage point.

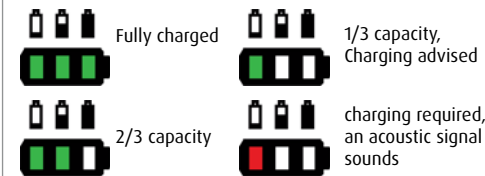

**Folding up lying surface**

The lying surface can easily be folded up to save space.

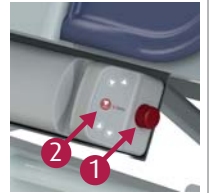

**Charging battery**

Charging the battery before it is completely empty will extend the lifespan.

Battery indication on hand control.


**Emergency stop & lowering button**

1. Operate the emergency stop in an emergency, if the upward or downward movement must be stopped immediately.
2. By pressing the lowering button in an emergency, the shower trolley will return to the lowest position.


**Cleaning and maintenance**

The Sirocco basic / Sirocco deluxe must be cleaned after every use, with water and regular household cleaning agents. Do not use any abrasive or other aggressive cleaning products. If necessary, the mattress can be disinfected with a solution of 70% ethanol or propanol. The complete maintenance schedule and cleaning protocol can be found in the instructions for use. ([www.lopital.com](http://www.lopital.com))

According to the regulation for medical devices, a Showerstretcher must be maintained in line with the manufacturer's instructions. Those instructions can be found in the maintenance protocol, see website Lopital ([www.lopital.com](http://www.lopital.com)).

In the event of a breakdown, contact your Lopital distributor.

**Instructions for use**  
For the full Instructions for use go to [www.lopital.com](http://www.lopital.com)


**Important!**

This quick user manual may not be considered a replacement for the instructions for use.