

Z:\SW-Tek\Sirocco 2400\2400\

Signed	PS	Date	21-09-2005
Revised	6 /ES	Date	16-11-2010
Revised	7 /KA	Date	29-01-2015

Overview parts Sirocco

Copyright reserved by law
Unit of measurement : mm

Lopital Nederland bv

Type nr: 61002400 / 2401

Sirocco 61002400 / 61002401

Part nr.	QTY.	Art.nr.	Part
1	1	70240004	Frame
2	2	70240011	Spacer
3	1	70240006	Cover plate
4	1	70210044	Topplate
5	1	70210019	Hook for hose
6	1	70240001	Mounting plate battery
7	1	70210511	Mounting plate
8	1	70210515	Control box
9	1	70210518	Batterypack
10	1	70210510	Manual control
11	2	70210023	Mattress hook
12	2	70210025	Clamping plate
13	1	70210038	Clamping plate (set)
14	1	70210024	Siderail
15	1	70240003	Siderail locked
16	1	70210047	Mattress
17	1	70210048	Pillow
18	1	70210055	Outlet
19	1	70210043	Clamping disk
20	1	70210056	Clamping ring
21	1	70210054	Hose (complete)
22	1	70210039	Lock
23	1	70210041	Leaf spring
24	2	70210052	Buffer rubber
25	1		Cover fastener siderail
26	1		Cylinder head screw
27	2		Ring
28	2		Tube cap
29	2		Slide bearing
30	2		Slide bearing
31	2		Round head screw
32	5		Locknut
33	5		Washer
34	4		Lock nut
35	4		Bolt
36	3		Hex bolt
37	8		Raised countersunk screw
38	2		Countersunk screw
39	2		Washer
40	2		Cap nut
41	1		Washer

Z:\SW-Tek\Sirocco 2400\2400\

Signed	PS	Date	21-09-2005
Revised	6 /ES	Date	16-11-2010
Revised	7 /KA	Date	29-01-2015
Copyright reserved by law	Unit of measurement : mm		


Overview parts Sirocco

Lopital Nederland bv

Type nr: 61002400/2401

Sirocco 61002400 / 61002401

Part nr.	QTY.	Art.nr.	Part
42	4		Washer
43	1		Hex bolt
44	1		Locknut
45	2		Nut cap
46	4		Hex bolt
47	2		Raised countersunk head screw
48	12		Bolt
49	2		Tube cap
50	12		Countersunk screw

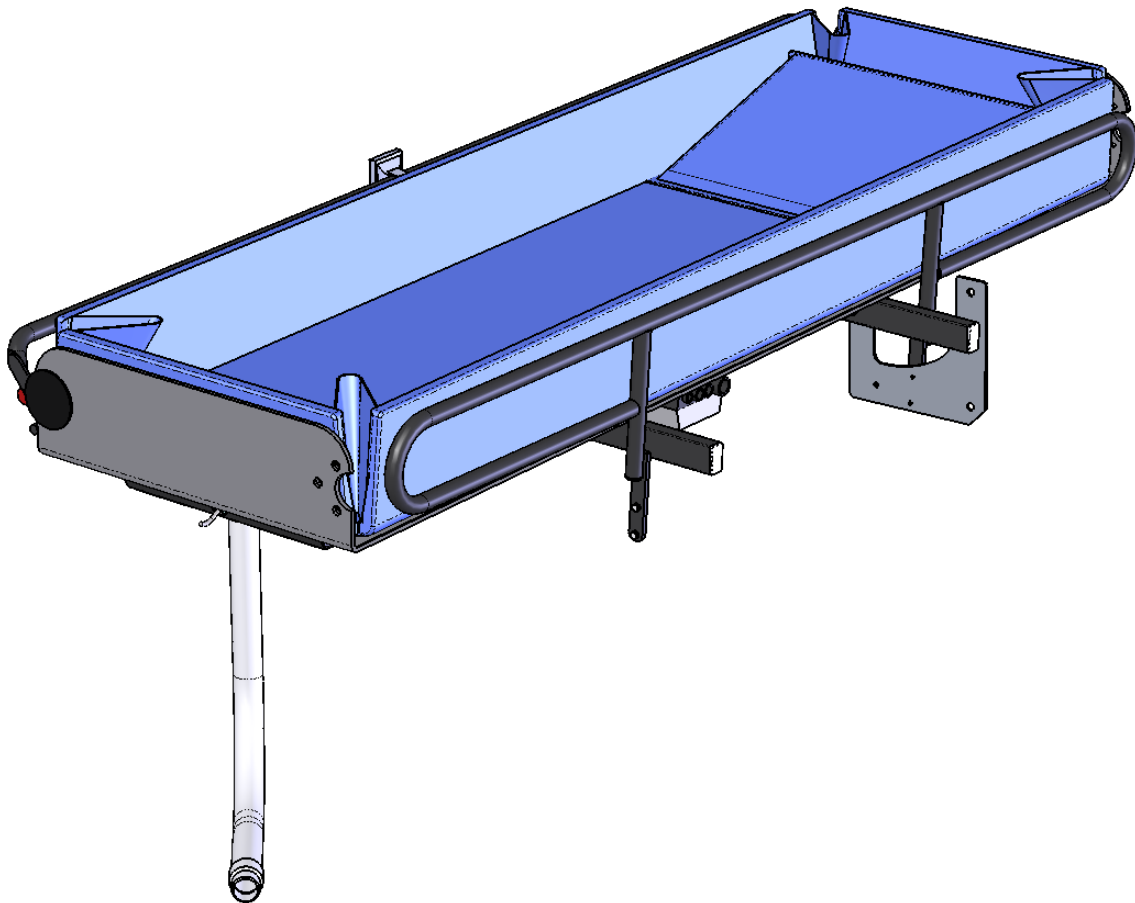


Z:\SW-Tek\Sirocco 2400\2400\

Signed	PS	Date	21-09-2005
Revised	6 /ES	Date	16-11-2010
Revised	7 /KA	Date	29-01-2015
Copyright reserved by law	Unit of measurement : mm		



Overview parts Sirocco

Lopital Nederland bv

Type nr: 61002400/2401

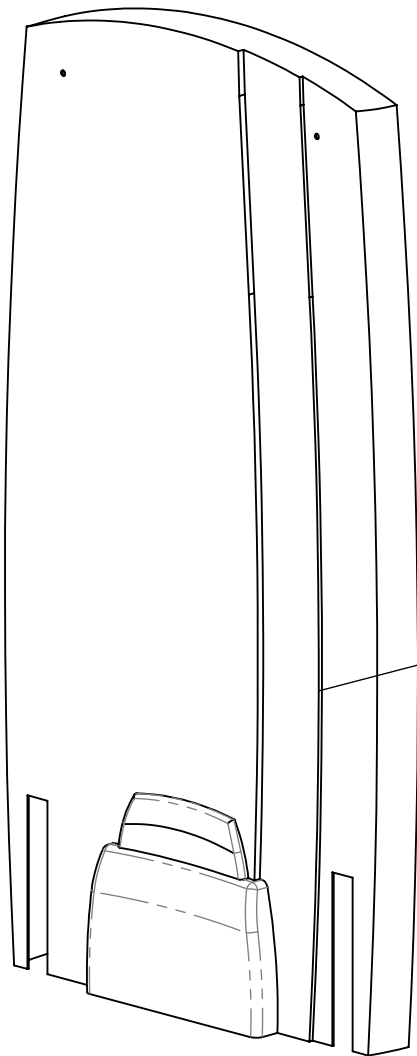

Place the charger outside the bathroom

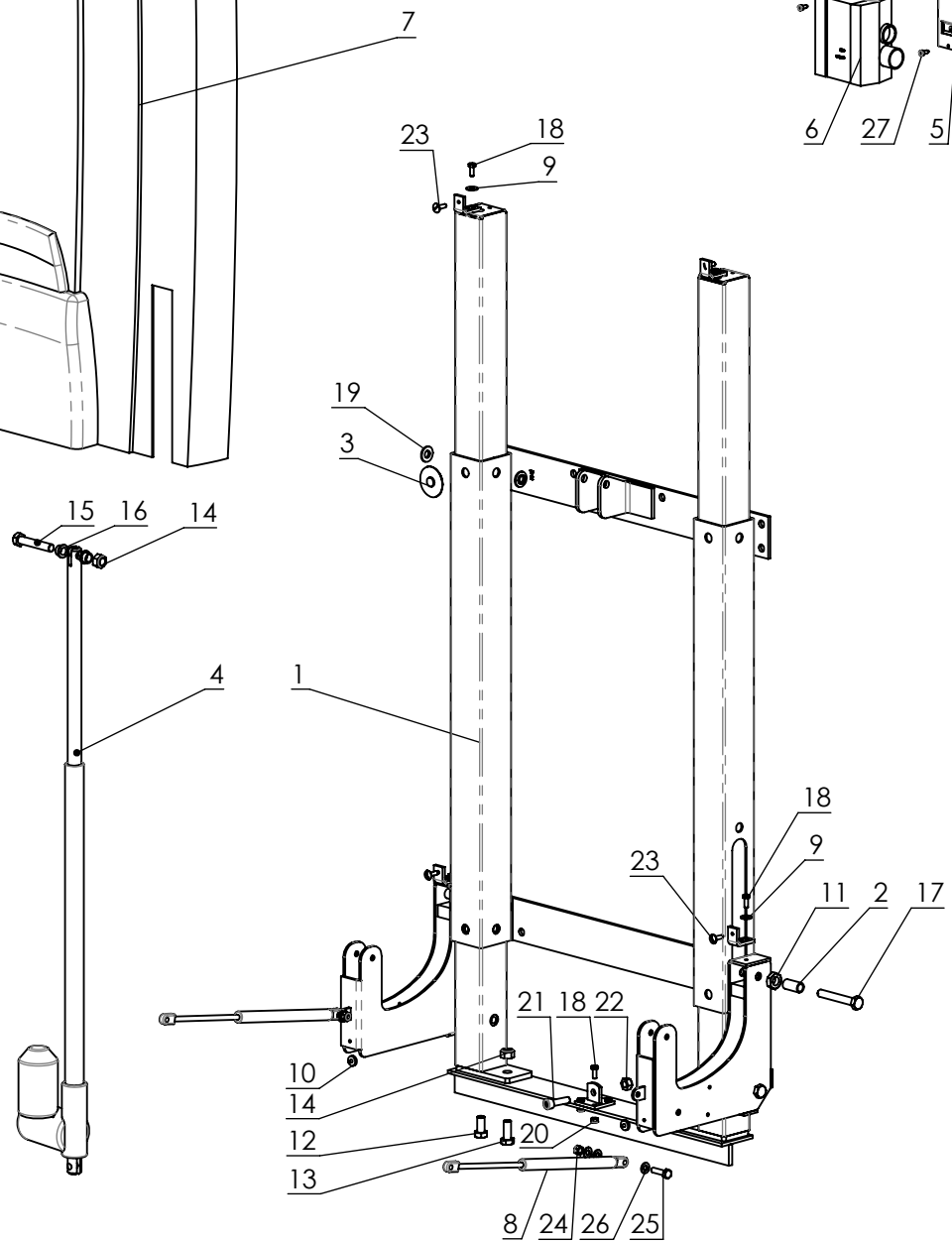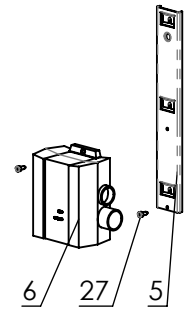

Z:\SW-Tek\Peilkaan lijn\5004\

Signed	PS	Date	11-07-2005
Revised	5 /PS	Date	16-12-2008
Revised	6 /KA	Date	15-01-2015

Overview parts Wallmount

Copyright reserved by law


Unit of measurement: mm

Lopital Nederland bv

Type nr: 5004

Adjusting high / low unit wall

Part nr.	QTY.	Art.nr.	Part
1	1	70240007	Wall frame Set complete
2	4	70240009	Spacer
3	4	70240008	Slide prop
4	1	70240002	Actuator
5	1	70210512	Mountingplate wall charger
6	1	70210516	Charger
7	1	70240010	Cover
8	2	70210040	Gas spring
9	8		Ring
10	4		Slider flatpin
11	4		Low Hex nut
12	2		Hex bolt
13	2		Hex bolt
14	3		Lock nut
15	1		Hex bolt
16	2		Slide bearing
17	4		Hex bolt
18	8		Hex bolt
19	10		bearing
20	4		Lock nut
21	1		Cylinder head screw
22	1		Locknut
23	4		Low button head screw
24	2		Locknut
25	2		Hex bolt
26	6		Washer
27	2		Cylinder head screw

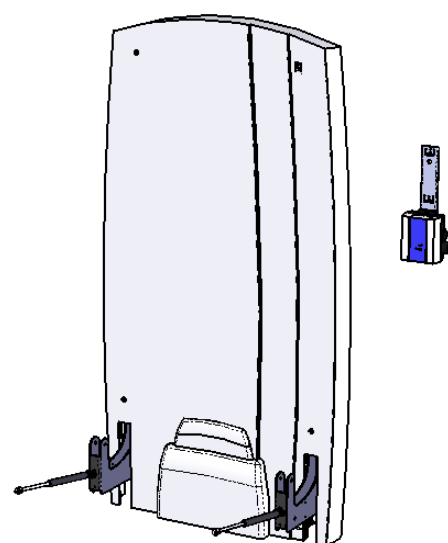

Z:\SW-Tek\Peilkaan lijn\5004\

Signed	PS	Date	11-07-2005
Revised	5 /PS	Date	16-12-2008
Revised	6 /KA	Date	15-01-2015

Overview parts Wallmount

Copyright reserved
by law

Unit of measure-
ment: mm

Lopital Nederland bv

Type nr: 5004