

Flexo shower toilet chair

Instruction card

01. Safety recommendations

- Read instruction for use before use (www.lopital.com).
- Maximum load 150 kg / 330 lbs.
- Make sure that the chair is on the brake during client transfers from and to the device.
- Place both feet on the foot supports.
- Front closure is closed during up / down movements and arms should rest on armrest.
- Only relocate and move the chair in the lower position.




02. Hydraulic operation

Up



Down

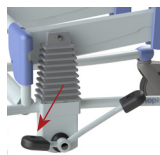


03. Use

1. Place the chair in the right position and fold the foot plates up.
2. Apply the brakes to the chair!
3. Open the armrests and fold these up if necessary.
4. Support the client when they are taking a seat.
5. Shut the armrests and let the client's feet rest on the footplates.

Stand up by carrying out the same procedure in reverse order.

04. Wheel blocking

1. Wheel blocking
2. Unlocking

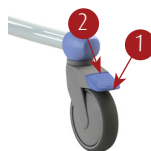


5100 5199
Flexo



05. Footrest

Both foot plates must be folded away before the person with a disability takes a seat in the chair. To do so, fold up the foot plates (1) or turn out both supports (2). Ensure that the client's feet are resting on both footplates while moving the chair.

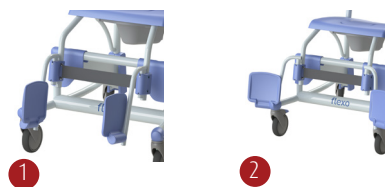

06. Armrest with front closure

1. Unlocking
2. Rotating
3. Folding up



07. Toilet function

- Remove the lid on the seat before the client sits down in the chair.
- *In combination with toilet:* the Flexo can be moved over almost every toilet. Move the chair upwards or downwards to achieve the right position.
- *With bedpan:* the Flexo can also be used as a toilet chair. Simply slide the stainless steel / plastic bedpan under the chair using the guides.

08. Cleaning and maintenance

The Flexo must be cleaned after every use, with water and regular household cleaning agents. Do not use any abrasive or other aggressive cleaning products. If necessary, the PUR components can be disinfected with a solution of 70% ethanol or propanol. The complete maintenance schedule and cleaning protocol can be found in the instructions for use (www.lopital.com).

According to the regulation MDR (EU) 2017/745 for medical devices, a chair must be maintained in line with the manufacturer's instructions. Those instructions can be found in the maintenance protocol, see website Lopital (www.lopital.com).

Important!

This quick instruction card may not be considered a replacement for the instruction for use!

In the event of a breakdown, contact Lopital.

Tel: +31 (0) 13-5239300

E-mail: info@lopital.nl

Instruction for use
For full instruction for use go to: www.lopital.com



 **ISO CERTIFIED**
9001 - 13485 - 14001

INS5199-2026-01 EN