
Safety recommendations


- Read the instructions for use carefully (find this at: www.lopital.com) before use.
- Maximum load 150 kg.
- Make sure that the chair is on the brake during client transfers from and to the device.
- Place both feet on the foot supports.
- Front closure is closed during up / down movements and arms should rest on armrest.
- Only relocate and move the chair in the lower position.

Operation

Up

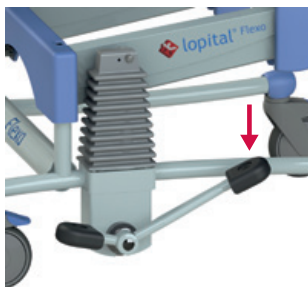Down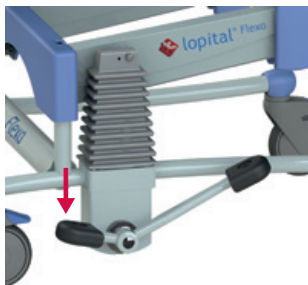

Important!

This instruction card may not be considered a replacement for the instructions for use.


Use

1. Place the chair in the right position and fold the foot plates up.
2. Apply the brake to the chair!
3. Open the armrests and fold these up if necessary.
4. Support the less able person when they are taking a seat.
5. Shut the armrests and let the client's feet rest on the foot plates.

Stand up by carrying out the same procedure (back to front).

Wheel blocking

1. Wheel blocking.
2. Unlocking.

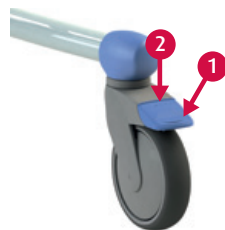
Foot rest

Both foot plates must be folded away for the less able person to take a seat in the chair. To do so, fold up the foot plates (1) or turn out both supports (2). Make sure that the less able person has both feet on the foot plates when moving.

Armrest with front closure

1. Unlocking.
2. Rotating.
3. Folding up.

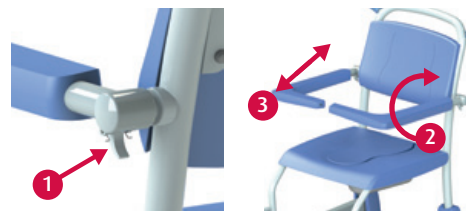Toilet function

- Remove the lid on the seat before the person sits down in the chair.
- *In combination with toilet:* The Flexo can be moved over almost every toilet. Move the chair upwards or downwards to achieve the right position.
- *With bedpan:* The Flexo can also be used as a toilet chair. Simply slide the stainless steel / plastic bedpan under the chair using the guides.

Cleaning and maintenance

The Flexo must be cleaned after every use, with water and regular household cleaning agents. Do not use any abrasive or other aggressive cleaning products. Check the wheels regularly for hairs, dust and soap residues and remove when necessary.

The complete maintenance schedule can be found in the instructions for use (**find this at: www.lopital.com**).

In the event of a breakdown, contact Lopital.
Tel: +31 (0)13 - 5239300, **Fax:** +31 (0)13 - 5239301
E-mail: service@lopital.nl

Instructions for use

For the full instructions for use go to **www.lopital.com**

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