



If caregivers have a choice, they choose Lopital



Periodical preventative
maintenance and inspection

Tango XXL Shower-toilechair

Tango XXL Shower-Toilechair

51005800



Important!

The Tango XXL must be serviced and inspected at least once a year.

Service and maintenance of the Tango XXL is only permitted by qualified persons and using original spare parts.

Making changes to the construction could affect the safety of the Tango XXL. This will also lead to Lopital's liability and warranty conditions being invalidated. As a result, the Tango XXL will no longer comply with the regulation MDR (EU) 217/745 for medical devices.

Each inspection or repair must be entered to the log.

The average technical life of the Tango XXL is 10 years, assuming it is cleaned daily and regularly maintained supplied.

Manufacturer:

Lopital Nederland B.V.
Laarakkerweg 9, 5061 JR Oisterwijk, the Netherlands
T +31 (0)13 5239300
www.lopital.com, info@lopital.nl

Index

1. General condition of the chair.....	4
2. Wheels.....	5
3. Armrests.....	7
4. Footrest.....	8

1. General condition of the chair
2. Wheels
3. Armrest
4. footrest

C= Check

S= Service/lubricate

A= Replacement advised

V= Replacement



Month	12	24	36	48	60	72	84	96	108	120
General condition	C	C	C	C	C	C	C	C	C	C
PUR	C	C	C	C	A	C	C	C	C	A
Wheels	C	C	C	C	C	C	C	C	C	C
Armrest	C	C	C	C	C	C	C	C	C	C
Footrest	C	C	C	C	A	C	C	C	C	A

1. General condition of the chair

Month	12	24	36	48	60	72	84	96	108	120
	C	C	C	C	A	C	C	C	C	A

Required tools

- Touch- up pen Sikkens A77A2

- Visually check if all labels are present and legible.
 - Serialnumberlabel:
 - Is the serial number still legible?
 - Is the datamatrix still readable for a scanner?
 - Is the CE logo still visible?
 - Inspectionlabel:
 - Is the inspection date still legible?
 - QR label:
 - Is the QR code still readable for a QR scanner?
- Visually check all painted parts for damage and corrosion. Touch up if necessary.
- Check PUR parts for severe discolouration (Brown in particular) and damage.
- Are all the screws present and securely fitted? Also that of the guide?

Present:

- Footrests (not a failure item)

For reasons of hygiene, it is strongly recommended to replace the seat and back once every 5 years.

2. Wheels

Month	12	24	36	48	60	72	84	96	108	120
	C	C	C	C	C	C	C	C	C	C

Required tools

- Allen key 8mm

New version: From serialnumber 5800.18.050



Figure 1 Wheel

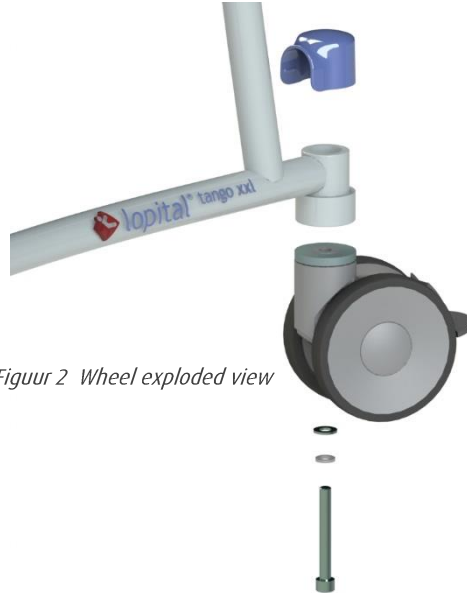


Figure 2 Wheel exploded view

- Check the attachment of the wheel (Tolerance)
- Check freewheel function + brakes.
- Do all four the wheels touch the floor on a flat surface? If not, check the chair right course.
- Brake lip intact?
- Check the tread.

Replace the wheels if they are damaged or worn.

The serial numbers 5800.18.030 to 5800.18.050. Are mounted with the new wheels (double roll) and an adapter set. These chairs still use the old frame (5800-11.8)

Old version: Before serialnumber 5800.18.030



Figure 3 Wheel

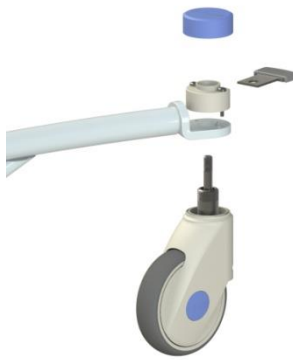


Figure 4 wheel exploded view

- Check the attachment of the wheel (Tolerance)
- Check freewheel function + brakes.
- Do all four the wheels touch the floor on a flat surface? If not, check the chair right course.
- Brake knob intact?
- Unlocking lip intact?
- Check the tread.

Replace the wheels if they are damaged or worn.

3. Armrests

Month	12	24	36	48	60	72	84	96	108	120
	C	C	C	C	C	C	C	C	C	C

Required tools

- Allen key 6mm

- Inspect its functioning, and check for any damage, corrosion. Always treat corrosion. Check its room for motion, measured at the front point.
- Are the armrests evenly positioned?
- Fastly secure screws.

Look for an exploded view on www.lopital.com



4. Footrest

Month	12	24	36	48	60	72	84	96	108	120
	C	C	C	C	A	C	C	C	C	A

Figure 5 Armrest
Required tools

-

New version: From serialnumber 5800.20.007

Chairs from this serial number are equipped with a height-adjustable footrest.

- Check for any damage.
- Are the foot plates evenly fitted and slanting upwards a little towards the middle?
- Check the height-adjustment, check for any damage.



Figure 6 Footrest

It is strongly recommended to replace the footplates every 5 years, due to hygiene.

Older version: From serialnumber 5800.20.007

- Check for any damage.
- Fold up foot plates (should be stiff and stay upright). (They should remain in the folded position, without falling back automatically)
- Are the foot plates evenly fitted and slanting upwards a little towards the middle?



Figure 7 Footrest

For questions about the maintenance protocol, contact Lopital (telephone +31 (0)13 5239300 or mail info@lopital.nl). You will need the serial number (outlined in red) to inform Lopital. See the serial number label on the product for this number.

 lopital® Laarakkerweg 9 5061 JR Oisterwijk The Netherlands T: (031)13-5239300	MD		
	REF		
	SN		
	MAX LOAD --- KG/---LBS		
IPX- Weight: -- kg Type battery: --- Drives operates: -- V Charge: ---V--- mA Non continuous: -----		CE Revision -	(01)----- (11)----- (21)---,---,---



If caregivers have a choice, they choose Lopital



For more information, visit our website: www.lopital.com

Lopital Nederland B.V. Laarakkerweg 9, 5061 JR Oisterwijk, Postbus 56, 5060 AB Oisterwijk
Tel +31 (0)13 5239300, E-mail info@lopital.nl, www.lopital.com