

# Marina shower trolley

## Instruction card

### 01. Safety recommendations


- Read the instruction for use before use ([www.lopital.com](http://www.lopital.com)).
- Maximum load 227 kg / 500 lbs.
- Make sure that the Marina Basic / Marina Deluxe is on the brake during client transfers from and to the shower trolley.




### 02. Manual control (electric)



### 03. Operation (hydraulic)



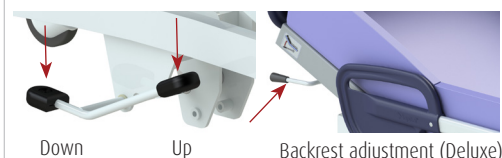


### 04. Tilting (Anti-Trendelenburg)

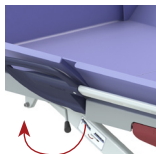


### 05. Wheel blocking Marina Basic (6100 2000/2050)

1. Wheel blocking
2. Unlocking

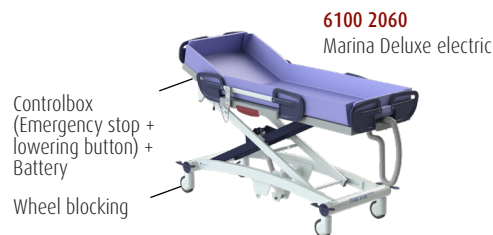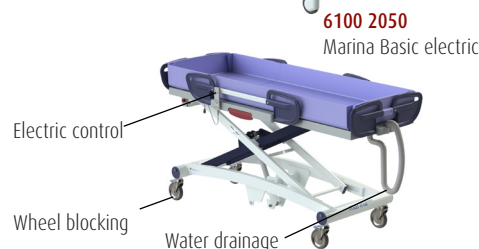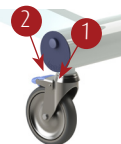


### 06. Wheel blocking Marina Deluxe (6100 2010/2060)

1. Wheel blocking (inside lever down)
2. Forward movement (footend) (outside lever down)
3. Horizontal setting: brakes release

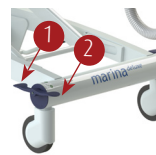

### 07. Side rail

1. Unlock (pull red handle (1))
2. Rotate the side rail to the desired position: closed / wide / open.

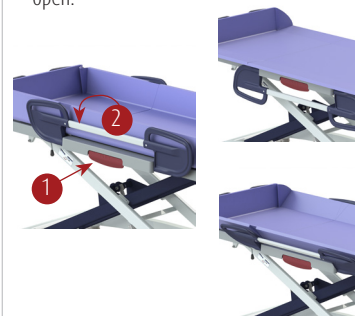

### 08. Water drainage

Water is drained through the drainage hose. To do this, remove the hose from the recess (1) and direct it to the lowest available drainage point. Always click the hose into the recess of the lying plate during transport.



### 09. Charging battery (electric)

Charging the battery before it is completely empty will extend the lifespan. Battery indicator is located on the hand control. The Marina can't be operated while the battery is charging.



fully charged



2/3 capacity




1/3 capacity, charging advised



charging required, an acoustic signal sounds

### 10. Emergency stop & lowering button (electric)

1. Activate the emergency stop in an emergency, if the upward or downward movement must be stopped immediately.
2. By pressing the lowering button in an emergency, the shower trolley will return to the lowest position.

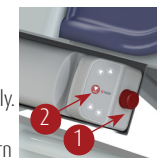

### 11. Cleaning and maintenance

The Marina Basic / Marina Deluxe must be cleaned after every use, with water and regular household cleaning agents. Do not use any abrasive or other aggressive cleaning products. If necessary, the mattress can be disinfected with a solution of 70% ethanol or propanol. The complete maintenance schedule and cleaning protocol can be found in the instructions for use ([www.lopital.com](http://www.lopital.com)).

According to the regulation MDR (EU) 2017/745 for medical devices, the Marina Basic & Marina Deluxe must be maintained in line with the manufacturer's instructions. Those instructions can be found in the maintenance protocol, see website Lopital ([www.lopital.com](http://www.lopital.com)).

#### Important!

This quick instruction for use may not be considered a replacement for the instruction for use!

In the event of a breakdown, contact Lopital.  
Tel: +31 (0) 13-5239300  
E-mail: [info@lopital.nl](mailto:info@lopital.nl)

Instruction for use  
For the full Instruction for use go to: [www.lopital.com](http://www.lopital.com)



ISO CERTIFIED  
9001 - 13485 - 14001

INS2000-2026-01 EN