

Nemo shower toilet chair

Instruction card

01. Safety recommendations

- Read instruction for use before use (www.lopital.com).
- Maximum load 150 kg / 330 lbs.
- Make sure that the chair is on the brake during client transfers from and to the device.
- Place both feet on the foot supports.
- Only drive and manoeuvre the Nemo in a non-tilted position.
- When moving the Nemo, do not tilt.



02. Operation


The Nemo is equipped with a function for tilting back to a maximum 30°.

1. Apply the brake to the chair.
2. Stand to the side of the Nemo and grip the handle in your right hand (1).
3. Hold the frame of the main support with your left hand (2).
4. The chair frame can be tilted by pressing in the operator handle.



5100 5199

Nemo

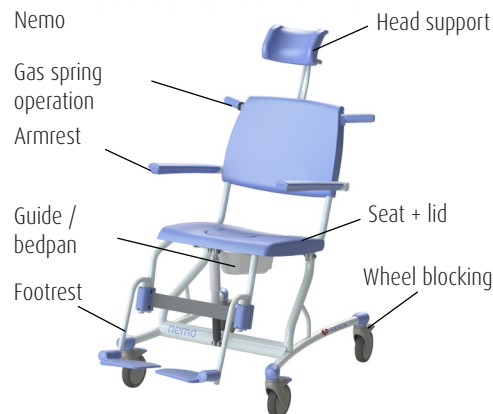


03. Use

1. Place the chair in the right position and turn the foot rests away.
2. Apply the brakes to the chair!
3. If necessary, fold the armrests up.
4. Support the client when they are taking a seat.
5. Shut the armrests and let the client's feet rest on the footplates.

Stand up by carrying out the same procedure in reverse order.

04. Wheel blocking

1. Wheel blocking
2. Unlocking

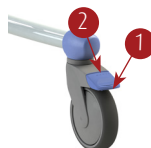


05. Footrest

Both foot plates must be folded away for the client to take a seat in the chair. To do so, fold up the foot plates (1) or turn out both supports (2). Make sure that the client has both feet on the foot plates when moving.



06. Armrest

1. Folding up

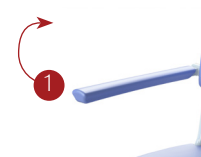

07. Toilet function

- Remove the lid on the seat before the client sits down in the chair.
- *In combination with toilet:* the Nemo can be moved over almost every toilet.
- *With bedpan:* the Nemo can also be used as a toilet chair. Simply slide the stainless steel / plastic bedpan under the chair using the guides.

11. Cleaning and maintenance

The Nemo must be cleaned after every use, with water and regular household cleaning agents. Do not use any abrasive or other aggressive cleaning products. If necessary, the PUR components can be disinfected with a solution of 70% ethanol or propanol. The complete maintenance schedule and cleaning protocol can be found in the instructions for use (www.lopital.com).

According to the regulation MDR (EU) 2017/745 for medical devices, a chair must be maintained in line with the manufacturer's instructions. Those instructions can be found in the maintenance protocol, see website Lopital (www.lopital.com).

Important!

This quick instruction card may not be considered a replacement for the instruction for use!

In the event of a breakdown, contact Lopital.

Tel: +31 (0) 13-5239300

E-mail: info@lopital.nl

Instruction for use
For the full instruction for use go to: www.lopital.com

