

Lotus active standup aid

Instruction card

01. Safety recommendations

- Read instruction for use before use (www.lopital.com).
- Maximum load 200 kg / 440 lbs (Lotus /Lotus Compact) or 227 kg / 500 lbs (Lotus XL)
- Make sure that the Lotus/Lotus Compact/Lotus XL is on the brake during client transfers from and to the device.





02. Use

1. Move the Lotus/Lotus Compact/Lotus XL to the client and turn away the seat parts.
2. If necessary adjust the legs of the Lotus/Lotus compact/Lotus XL and castor it as closely as possible to the client.
3. Apply the brake to the Lotus/Lotus Compact/Lotus XL.
4. Allow the client to stand up and turn seat part back.
5. The client can now sit down.
Stand up by carrying out the same procedure in reverse order.

03. Castor blocking

1. Castor blocking
2. Unlocking

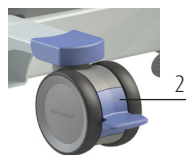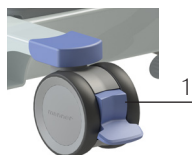

5900 8200
Lotus



5900 8250
Lotus Compact




5900 8270
Lotus XL



04. Base spreader


The Lotus/Lotus Compact/Lotus XL is equipped with a mechanical leg spreader.

By pressing 1 side of the pedal, the legs move inward or outward (depending on which side of the pedal is pressed).

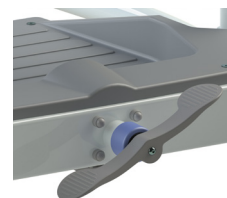


05. Rotatable seat parts

The Lotus/Lotus Compact is equipped with two comfortable seats that can be swiveled away. The Lotus/Lotus Compact has an entry of 44 cm / 17.3 inch, when the seat parts are turned away.

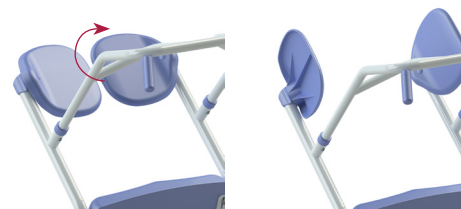

The Lotus XL has an entry of 52 cm / 20.5 inch.

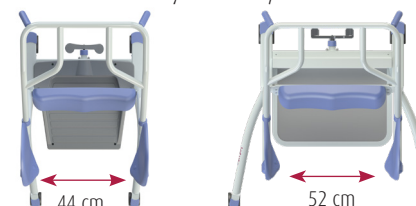

06. Adjustable leg support

By loosening the nuts that secure the lower leg support, the lower leg support can be adjusted up or down.



07. Cleaning and maintenance

The Lotus/Lotus Compact/Lotus XL must be cleaned after every use, with water and regular household cleaning agents. Do not use any abrasive or other aggressive cleaning products. If necessary, the PUR components can be disinfected with a solution of 70% ethanol or propanol. The complete maintenance schedule and cleaning protocol can be found in the instructions for use (www.lopital.com).

According to the regulation MDR (EU) 2017/745 for medical devices, the Lotus/Lotus Compact/Lotus XL must be maintained in line with the manufacturer's instructions. Those instructions can be found in the maintenance protocol, see website Lopital (www.lopital.com).

Important!

This quick instruction card may not be considered a replacement for the instruction for use!

In the event of a breakdown, contact Lopital.
Tel: +31 (0) 13-5239300
E-mail: info@lopital.nl

Instruction for use
For the full Instruction for use go to:
www.lopital.com

