

Reflex shower toilet chair

Instruction card

01. Safety recommendations

- Read instruction for use before use (www.lopital.com).
- Maximum load 135 kg / 300 lbs.
- Make sure that the chair is on the brake during client transfers from and to the device.
- Place both feet on the foot supports.
- Front closure is closed during up / down movements and arms should rest on armrest.
- Only relocate and move the chair in the lower position.



02. Operation

Up



Tilt forwards



Down



Tilt backwards




Activate (press both simultaneously) Lock button



03. Armrest with front closure

1. Unlocking
2. Rotating
3. Folding up

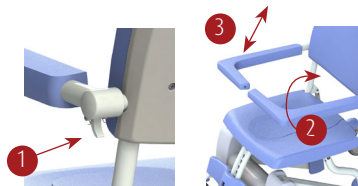

5100 5600
Reflex



Charger

5100 5605
Reflex with Flexo
leg supports




04. Use

1. Activate the Reflex by pressing both 'up' buttons simultaneously for 2 seconds. Note: After use, the Reflex will go into a sleep setting after 60 minutes.
2. Position the chair correctly and fold the footrests up.
3. Apply the brakes to the chair!
4. Open the armrests and fold these up if necessary.
5. Support the client when they are taking a seat.
6. Shut the armrests and let the client's feet rest on the footplates.

Stand up by carrying out the same procedure in reverse order. All functions on the Reflex are halted as a result of using the lock button. The Reflex must be reset. You do this by pressing both 'up' buttons for 4 seconds.

05. Wheel blocking

1. Wheel blocking
2. Unlocking

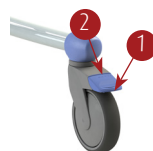

06. Footrest

1. Unlocking
2. Remove complete footrest
3. Choose desired position
4. Hook in and lock

Make sure that the client has both feet on the foot plates when moving.



1



2

07. Charging batteries

After using the Reflex, connect it to the charger to maintain the batteries.



Off
Green
Blinking orange
Red



Sleep setting
Chair ready to use
Battery is almost empty
Battery is empty


The Reflex will not work during charging.

08. Toilet function

- Remove the lid from the seat before the client with a disability takes a seat in the chair.
- *In combination with toilet:* the Reflex can be moved over almost every toilet. Move the chair upwards or downwards to achieve the right position.
- With bedpan: the Reflex can also be used as a toilet chair. Simply slide the stainless steel / plastic bedpan under the chair using the guides.

09. Emergency stop & lowering button

1. If all the functions fail, the Reflex can be lowered using the emergency lowering lever.
2. All functions of the Reflex can be disabled using the lock button. To reset the Reflex, press and hold both 'up' buttons simultaneously for 4 seconds.

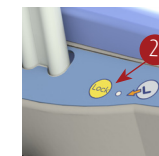

10. Cleaning and maintenance

The Reflex must be cleaned after every use, with water and regular household cleaning agents. Do not use any abrasive or other aggressive cleaning products. If necessary, the PUR components can be disinfected with a solution of 70% ethanol or propanol. The complete maintenance schedule and cleaning protocol can be found in the instructions for use (www.lopital.com).

According to the regulation MDR (EU) 2017/745 for medical devices, Reflex must be maintained in line with the manufacturer's instructions. Those instructions can be found in the maintenance protocol, see website Lopital (www.lopital.com).

Important!

This quick instruction for use may not be considered a replacement for the instruction for use!

In the event of a breakdown, contact Lopital.

Tel: +31 (0) 13-5239300

E-mail: info@lopital.nl

Instruction for use
For the full instruction for
use go to: www.lopital.com

