

Elexo shower toilet chair


Instruction card

01. Safety recommendations


- Read instruction for use before use (www.lopital.com).
- Maximum load 180 kg / 400 lbs.
- Make sure that the chair is on the brake during client transfers from and to the device.
- Place both feet on the foot supports.
- Front closure is closed during up / down movements and arms should rest on armrest.
- Only relocate and move the chair in the lower position.



02. Operation

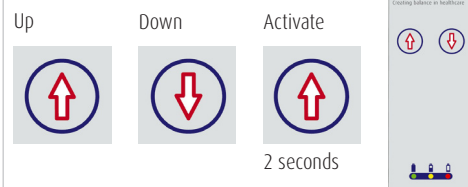

5100 5500
Elexo

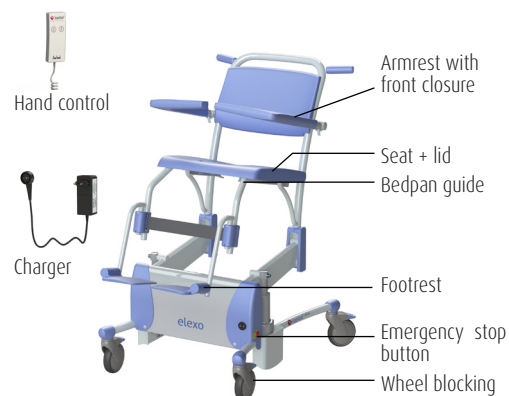


03. Use

1. Activate the Elexo by pressing the 'up' button for 2 seconds.
Note: After use, the Elexo will go into a sleep setting after 60 minutes.
2. Place the chair in the right position and fold the foot plates up.
3. Apply the brakes to the chair!
4. Open the armrests and fold these up if necessary.
5. Support the client when they are taking a seat.
6. Shut the armrests and let the client's feet rest on the footplates.

Stand up by carrying out the same procedure in reverse order.


04. Wheel blocking

1. Wheel blocking.
2. Unlocking.

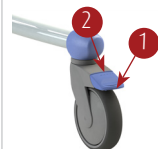

05. Footrest

Both foot plates must be folded away for the client to take a seat in the chair. To do so, fold up the foot plates (1) or turn out both supports (2). Make sure that the client has both feet on the foot plates when moving.

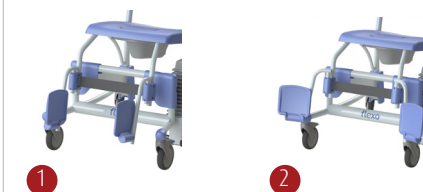

06. Charging batteries

After using the Elexo, connect it to the charger to maintain the batteries. The Elexo does not work while charging.




- | | |
|-----------------|--------------------------|
| Off | Sleep setting |
| Green | Chair ready to use |
| Blinking orange | Emergency stop activated |
| Orange | Battery is half empty |
| Blinking red | Battery is almost empty |
| Red | Battery is empty |

07. Toilet function

- Remove the lid on the seat before the client sits down in the chair.
- *In combination with toilet:* the Elexo can be moved over almost every toilet. Move the chair upwards or downwards to achieve the right position.
- *With bedpan:* the Elexo can also be used as a toilet chair. Simply slide the stainless steel / plastic bedpan under the chair using the guides.


08. Emergency stop & lowering button

1. Activate the emergency stop in the event of an emergency where the upward or downward movement must be stopped immediately.
2. When the lowering button is activated in an emergency, the chair lowers to its lowest position.

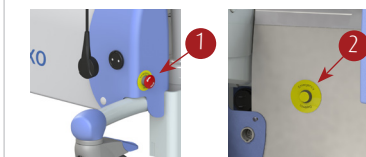

09. Armrest with front closure

1. Unlocking.
2. Rotating.
3. Folding up.

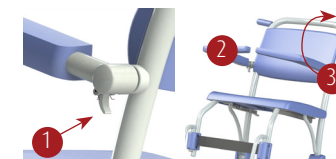

10. Cleaning and maintenance

The Elexo must be cleaned after every use, with water and regular household cleaning agents. Do not use any abrasive or other aggressive cleaning products. If necessary, the PUR components can be disinfected with a solution of 70% ethanol or propanol. The complete maintenance schedule and cleaning protocol can be found in the instructions for use (www.lopital.com).

According to the regulation MDR (EU) 2017/745 for medical devices, a chair must be maintained in line with the manufacturer's instructions. Those instructions can be found in the maintenance protocol, see website Lopital (www.lopital.com).

Important!

This quick instruction card may not be considered a replacement for instruction for use!

In the event of a breakdown, contact Lopital.
Tel: +31 (0) 13-5239300
E-mail: info@lopital.nl

Instruction for use
For the full instruction for use go to: www.lopital.com

