
Safety recommendations

- Read the instructions for use carefully (find this at: www.lopital.com) before use.
- Maximum load 180 kg.
- Make sure that the chair is on the brake during client transfers from and to the device.
- Place both feet on the foot support.
- Armrests are closed during up / down movements and arms should rest on armrest.
- Only relocate and move the chair in the lower position.

Operation panel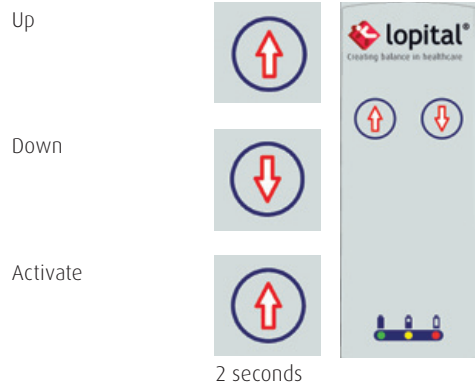

Important!

This instruction card may not be considered a replacement for the instructions for use.


Use

1. Activate the Elexo by pressing the 'up' button for 2 seconds.
Note: After use, the Elexo will go into a sleep setting after 60 minutes.
2. Place the chair in the right position and fold the foot supports up.
3. Apply the brake to the chair!
4. Open the armrests and fold these up if necessary.
5. Support the less able person when they are taking a seat.
6. Shut the armrests and let the client's feet rest on the foot plates.


Stand up by carrying out the same procedure in reverse order.

Wheel blocking

1. Wheel blocking.
2. Unlocking.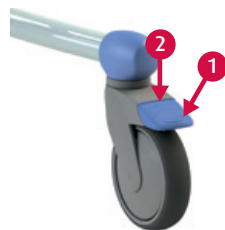


Foot rest

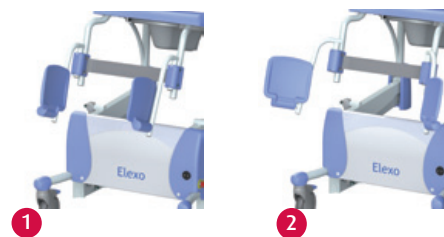Both foot plates must be turned away for the less able person to take a seat in the chair. Make sure that the person places both feet on the foot plates when the chair is moving.

Armrest with front closure

1. Unlocking.
2. Rotating.
3. Folding up.

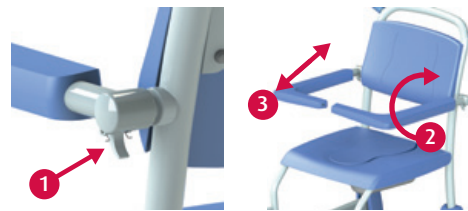Charging batteries

After using the Elexo, connect it to the charger to maintain the batteries.

- | | |
|-----------------|---------------------------|
| Off | Sleep setting. |
| Green | Chair ready to use. |
| Blinking orange | Emergency stop activated. |
| Orange | Battery is half empty. |
| Blinking red | Battery almost empty. |
| Red | Battery empty. |


The Elexo will not work during charging.

Toilet function

- Remove the lid on the seat before the person sits down in the chair.
- *In combination with toilet:* The Elexo can be placed over almost all toilets. Move the chair up or down to achieve the right position.
- *With bedpan:* The Elexo can also be used as a toilet chair. Simply slide the stainless steel / plastic bedpan under the chair using the guides.

Emergency lowering

If the Elexo breaks down, you can lower the chair using the emergency lowering button.

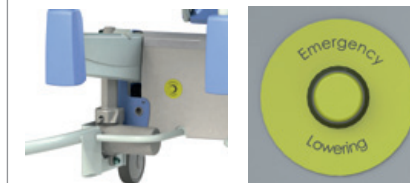Cleaning and maintenance

The Elexo must be cleaned after every use, with water and regular household cleaning agents. Do not use any abrasive or other aggressive cleaning products. Check the wheels regularly for hairs, dust and soap residues and remove when necessary.

The complete maintenance schedule can be found in the instructions for use (**find this at: www.lopital.com**).

In the event of a breakdown, contact Lopital.
Tel: +31 (0)13 - 5239300, **Fax:** +31 (0)13 - 5239301
E-mail: service@lopital.nl

Instructions for use

For the full instructions for use go to **www.lopital.com**

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