

Elexo XXL shower toilet chair


Instruction card

01. Safety recommendations


- Read instructions for use before use (www.lopital.com).
- Maximum load 220 kg.
- Make sure that the chair is on the brake during client transfers from and to the device.
- Place both feet on the foot supports.
- Armrests are closed during up / down movements and arms should rest on armrest.
- Only relocate and move the chair in the lower position.



02. Operation

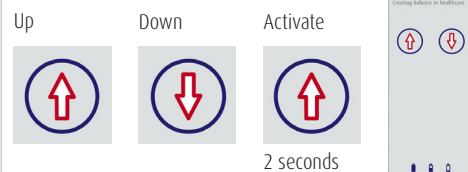

5100 5300
Elexo XXL

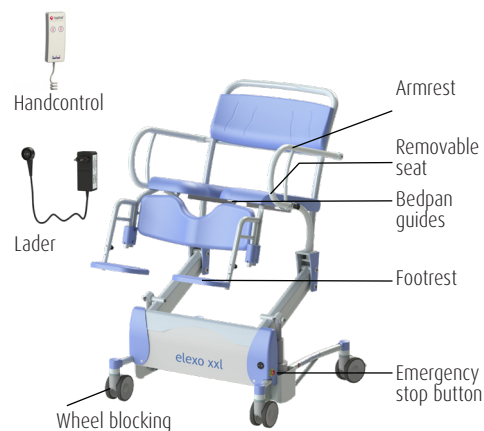

03. Use

1. Activate the Elexo XXL by pressing the 'up' button for 2 seconds. Note: After use, the Elexo XXL will go into a sleep setting after 60 minutes.
 2. Place the chair in the right position and fold the foot supports up.
 3. Apply the brake to the chair!
 4. Open the armrests and, if necessary, fold them upwards.
 5. Support the person with disability when they are taking a seat.
 6. Shut the armrests and let the client's feet rest on the foot plates.
- Stand up by carrying out the same procedure in reverse order.*


04. Wheel blocking

1. Wheel blocking.
 2. Unlocking.
- Directional wheel
1. Wheel blocking.
2. Unlocking.



05. Emergency stop & lowering button

1. Activate the emergency stop in the event of an emergency where the upward or downward movement must be stopped immediately.
2. When the lowering button is activated in an emergency, the chair lowers to its lowest position.

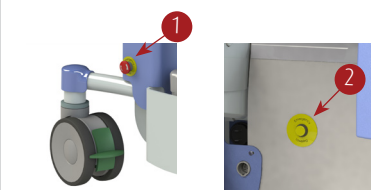

06. Charging batteries

After using the Elexo XXL, connect it to the charger to maintain the batteries. The Elexo XXL will not work during charging.



- | | |
|-----------------|--------------------------|
| Off | Sleep setting |
| Green | Chair ready to use |
| Blinking orange | Emergency stop activated |
| Orange | Battery is half empty |
| Blinking red | Battery almost empty |
| Red | Battery empty |

07. Removable seat



08. Armrest without front closure

1. Unlocking.
2. Turn backwards. Close by repeating process in reverse order.



09. Footrest

Both foot plates must be turned away for the less able person to take a seat in the chair. Make sure that the person places both feet on the foot plates when the chair is moving.



10. Toilet function

- *In combination with toilet:* the Elexo XXL can be moved over almost every toilet. Move the chair upwards or downwards to achieve the right position.
- *With bedpan:* the Elexo XXL can also be used as a toilet chair. Simply slide the stainless steel / plastic bedpan under the chair using the guides.

11. Cleaning and maintenance

The Elexo XXL must be cleaned after every use, with water and regular household cleaning agents. Do not use any abrasive or other aggressive cleaning products. If necessary, the mattress can be disinfected with a solution of 70% ethanol or propanol. The complete maintenance schedule and cleaning protocol can be found in the instruction for use (www.lopital.com).

According to the regulation MDR (EU) 2017/745 for medical devices, the Amfora/Stylex/Vienna must be maintained in line with the manufacturer's instructions. Those instructions can be found in the maintenance protocol, see website Lopital (www.lopital.com).

Important!

This instruction card may not be considered a replacement for the instruction for use!

In the event of a breakdown, contact Lopital.
Tel: +31 (0) 13-5239300
E-mail: info@lopital.nl

Instruction for use
For the full instruction for use go to: www.lopital.com

