



If caregivers have a choice, they choose Lopital



Periodical preventative
maintenance and inspection

Tango XL Shower-toilechair

Tango XL Shower-Toiletoiletchair

51005705



Important!

The Tango XL must be serviced and inspected at least once a year.

Service and maintenance of the Tango XL is only permitted by qualified persons and using original spare parts.

Making changes to the construction could affect the safety of the Tango XL. This will also lead to Lopital's liability and warranty conditions being invalidated. As a result, the Tango XL will no longer comply with the regulation MDR (EU) 217/745 for medical devices.

Each inspection or repair must be entered to the log.

The average technical life of the Tango XL is 10 years, assuming it is cleaned daily and regularly maintained supplied.

Manufacturer:

Lopital Nederland B.V.
Laarakkerweg 9, 5061 JR Oisterwijk, the Netherlands
T +31 (0)13 5239300
www.lopital.com, info@lopital.nl

Contents

1. General condition of the chair.....	4
2. Wheels.....	5
3. Arm rests.....	6
4. Footrest.....	7

1. General condition of the chair
2. Wheels
3. Armrest
4. footrest

C= Check

S= Service/lubricate

A= Replacement advised

V= Replacement



Month	12	24	36	48	60	72	84	96	108	120
General condition	C	C	C	C	C	C	C	C	C	C
PUR	C	C	C	C	A	C	C	C	C	A
Wheels	C	C	C	C	C	C	C	C	C	C
Armrest	C	V	C	V	C	V	C	V	C	V
Footrest	C	C	C	C	A	C	C	C	C	A

1. General condition of the chair

Month	12	24	36	48	60	72	84	96	108	120
	C	C	C	C	A	C	C	C	C	A

Required tools

- Touch- up pen Sikkens A77A2

- Visually check if all labels are present and legible.
 - Serialnumberlabel:
 - Is the serial number still legible?
 - Is the datamatrix still readable for a scanner?
 - Is the CE logo still visible?
 - Inspectionlabel:
 - Is the inspection date still legible?
 - QR label:
 - Is the QR code still readable for a QR scanner?
- Visually check all painted parts for damage and corrosion. Touch up if necessary.
- Check PUR parts for severe discolouration (Brown in particular) and damage.
- Are all the screws present and securely fitted? Also that of the guide?

Present:

- Footrest(not a failure item)
- 2 parts of de seat

For reasons of hygiene, it is strongly recommended to replace the seat and back once every 5 years.

2. Wheels

Month	12	24	36	48	60	72	84	96	108	120
	C	C	C	C	C	C	C	C	C	C

Required tools

- Allen key 8mm

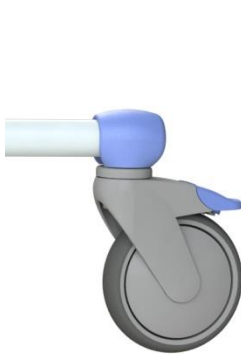


Figure 1 Wheel



Figure 2 wheel exploded

- Check the attachment of the wheel (Tolerance)
- Check freewheel function + brakes.
- Do all four the wheels touch the floor on a flat surface? If not, check the chair right course.
- Brake lip intact?
- Check the tread.
- Check the function of the directional wheel.

Replace the wheels if they are damaged or worn.

3. Arm rests

Month	12	24	36	48	60	72	84	96	108	120
	C	V	C	V	C	V	C	V	C	V

Required tools

- Allen key 6mm

- Inspect its functioning, and check for any damage, corrosion.
- Always treat corrosion.
- Are the armrests evenly positioned?
- Slide PUR back if necessary.
- The stainless-steel (A2) bolts should always be replaced by M8x12 DIN912 Zincflake.
- Fastly secure bolts. *(See figure 4)*

Replace bolts once every 2 years.

Look for an exploded view on www.lopital.com



Figure 3 Armrest

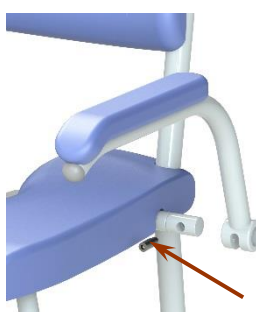


Figure 4 Screws

4. Footrest

Month	12	24	36	48	60	72	84	96	108	120
	C	C	C	C	C	C	C	C	C	C

Required tools

-

New version: From serialnumber 5705.20.001

Chairs from this serial number are equipped with a height-adjustable footrest.

- Check for any damage.
- Are the foot plates evenly fitted and slanting upwards a little towards the middle?
- Check the height-adjustment, check for any damage.



Figure 5 Footrests

It is strongly recommended to replace the footplates every 5 years, due to hygiene.

Older version: Before serialnumber 5705.20.001

- Check for any damage.
- Are the foot plates evenly fitted and slanting upwards a little towards the middle?

For reasons of hygiene, it is strongly recommended to replace the footplates once every 5 years.



Figure 6 Footrests

For questions about the maintenance protocol, contact Lopital (telephone +31 (0)13 5239300 or mail info@lopital.nl). You will need the serial number (outlined in red) to inform Lopital. See the serial number label on the product for this number.

 lopital®		MD -----	
 Laarakkerweg 9 5061 JR Oisterwijk The Netherlands T: (031)13-5239300		REF -----	
IPX- Weight: -- kg Type battery: --- Drives operates: -- V Charge: ---V--- mA Non continuous: -----		SN -----	
		 --- KG/---LBS  -----	(01)----- (11)----- (21)-----
			Revision -



If caregivers have a choice, they choose Lopital



For more information, visit our website: www.lopital.com

Lopital Nederland B.V. Laarakkerweg 9, 5061 JR Oisterwijk, Postbus 56, 5060 AB Oisterwijk
Tel +31 (0)13 5239300, E-mail info@lopital.nl, www.lopital.com

MAIN 5705-2024-01