

Vienna dressing table


Instruction card

01. Safety recommendations

- Read the instruction for use before use (www.lopital.com).
- Maximum load 227 kg / 500 lbs (Deluxe) and 150 kg / 330.7 lbs (Basic)




02. Manual control

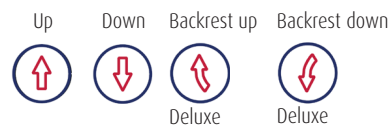


03. Side rail

- Unlock (pull handle (1))
- Rotate the side rail to the desired position: closed / wide / open (2).

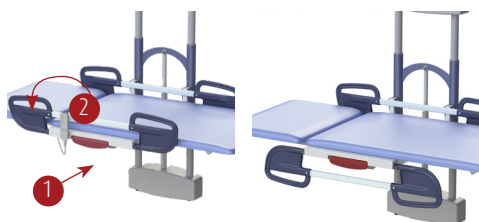

6500 5030
Vienna Deluxe

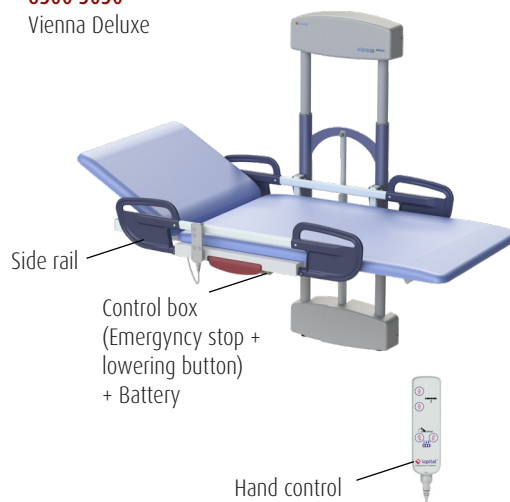

6500 5035
Vienna Basic



04. Folding up lying surface

The lying surface can easily be folded up to save space.



05. Charging battery

Charging the battery before it is completely empty will extend the lifespan.

Battery indicator is located on the hand control.




Fully charged



1/3 capacity, charging advised




2/3 capacity

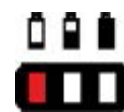

charging required, an acoustic signal sounds

06. Emergency stop & lowering button

- Activate the emergency stop in an emergency, if the upward or downward movement must be stopped immediately.

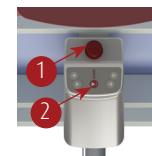

- By pressing the lowering button in an emergency, the shower trolley will return to the lowest position.

07. Cleaning and maintenance

The Vienna must be cleaned after every use, with water and regular household cleaning agents. Do not use any abrasive or other aggressive cleaning products. If necessary, the mattress can be disinfected with a solution of 70% ethanol or propanol. The complete maintenance schedule and cleaning protocol can be found in the instructions for use (www.lopital.com).

According to the regulation MDR (EU) 2017/745 for medical devices, the Vienna must be maintained in line with the manufacturer's instructions. Those instructions can be found in the maintenance protocol, see website Lopital (www.lopital.com).

Important!

This quick instruction card may not be considered a replacement for the instruction for use!

In the event of a breakdown, contact Lopital.

Tel: +31 (0) 13-5239300

E-mail: info@lopital.nl

Instruction for use
For the full instruction for use go to: www.lopital.com

