


From Serial number 5300.20.005

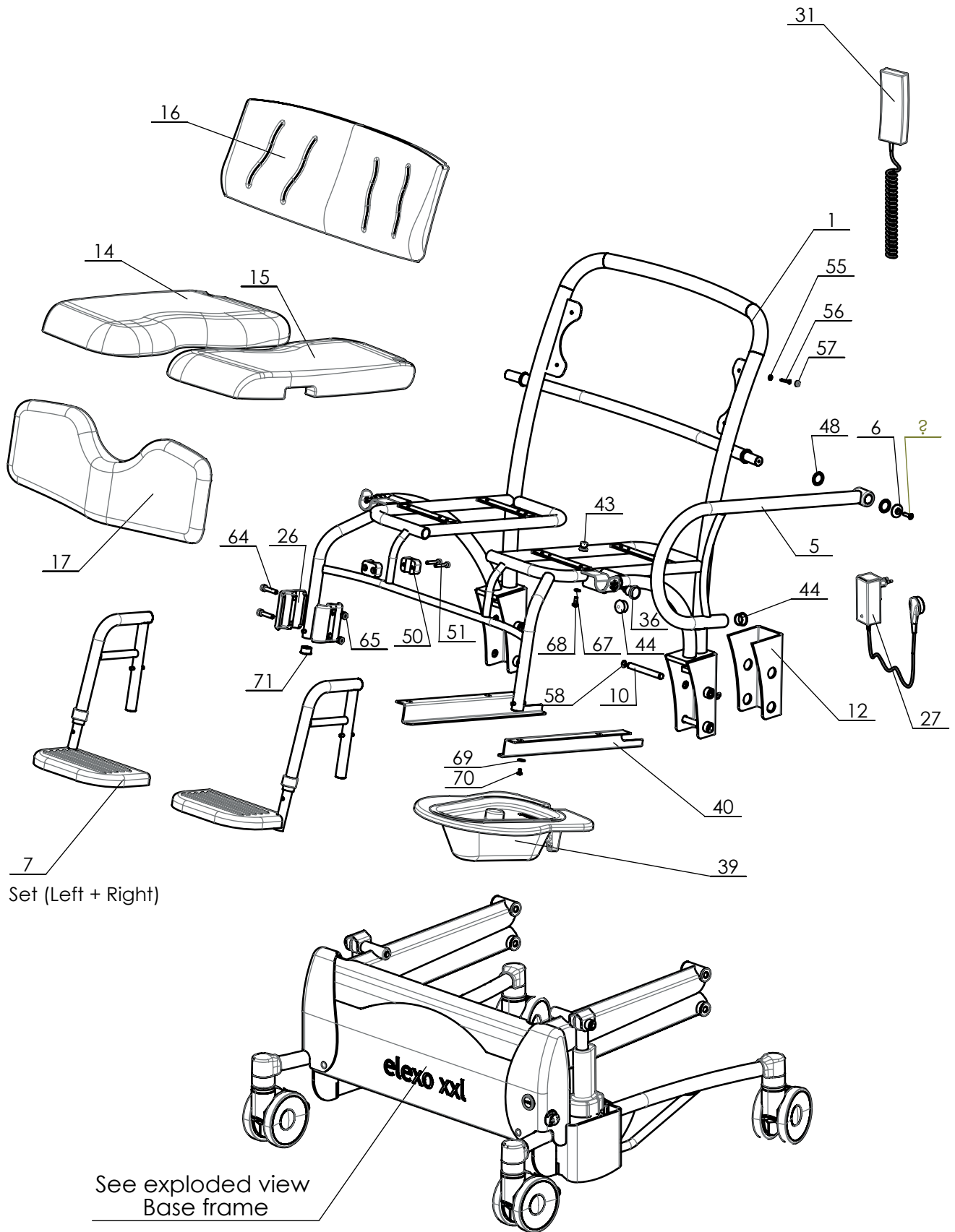

Set (Left + Right)

See exploded view  
Base frame

P:\SW-Tek\Elexo XXL 5300\

Drawn by: ES	Date: 10-10-2013
Revised: 6 /KA	Date: 18-01-2023
Revised: 7 /KA	Date: 29-09-2023


Copyright reserved  
by law

unit measure-  
ment: mm

# Overview parts Elexo XXL

Lopital Nederland bv

Type no: 51005300

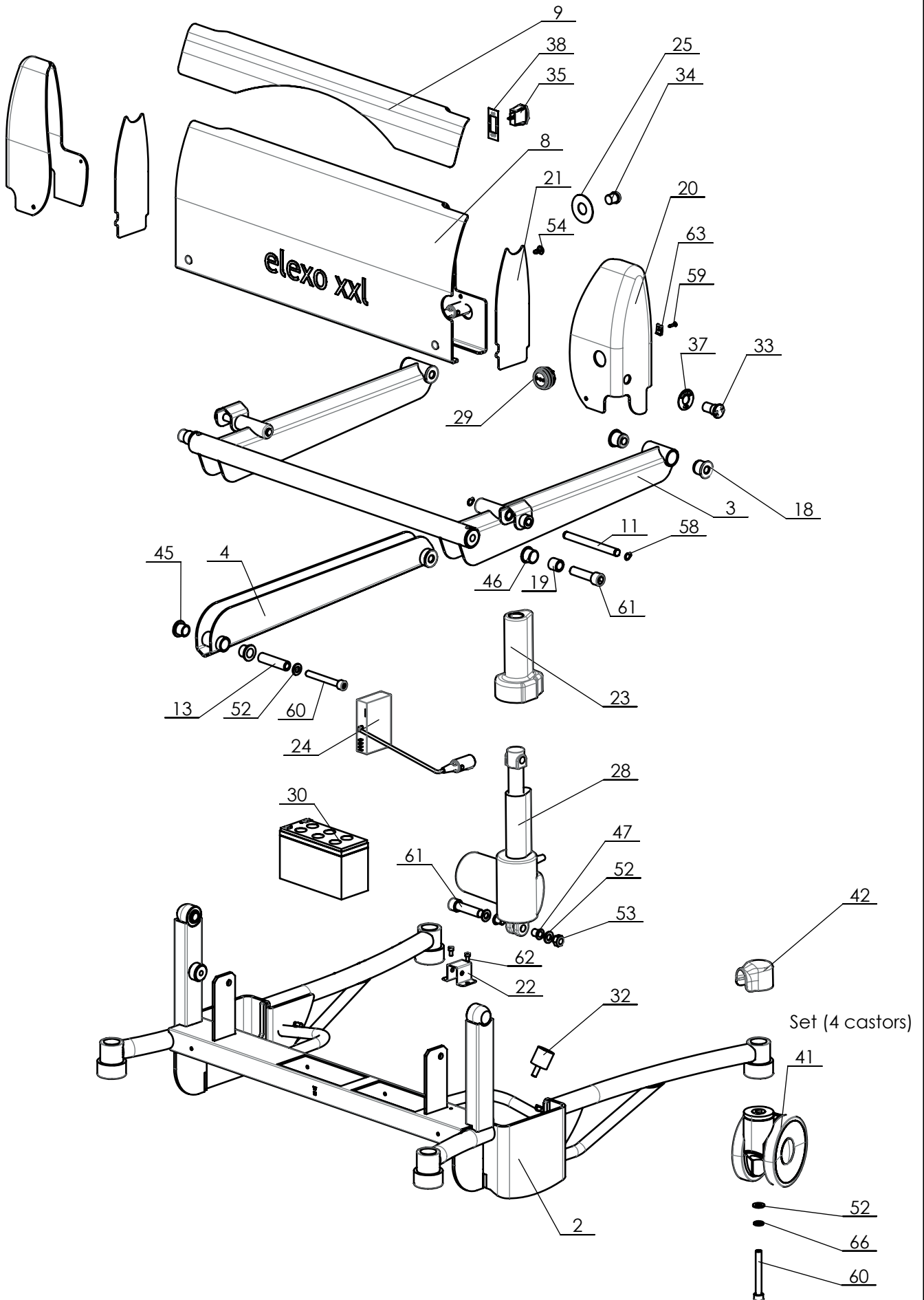

P:\SW-Tek\Eleko\Eleko XXL 5300\

Drawn by: ES	Date: 10-10-2013
Revised: 6 /KA	Date: 18-01-2023
Revised: 7 /KA	Date: 29-09-2023

# Overview parts Elexo XXL base frame

Copyright reserved by law

Unit of measurement: mm

Lopital Nederland bv

Type no: 51005300

## Elexo XXL 51005300

Part nr.	QTY.	Art.nr.	Part
1	1	70530004	Seat frame
2	1	70530001	Frame
3	1	70530002	Main lever
4	2	70530003	Control lever
5	1	70530005	Armrest (L+R)
6	2	75530017	Rosette
7	1	70530017	Height adjustable footsupport (Set)
8	1	70530018	Battery cover
9	1	70530019	sticker motorcover
10	4	70530007	Hinge pin L=100
11	2	70530016	Hinge pin L=127
12	2	70530027	Hinge cover blue
13	2	70530024	Bush L=63 mm
14	1	70530014	Seat XXL (Right)
15	1	70530008	Seat XXL (Left)
16	1	70530009	Backrest
17	1	70530012	Calf plate
18	8	70560041	Bush
19	2	70530011	Bush
20	1	70550011	Battery Cover sides (set)
21	2	70550019	Seperation plate
22	1	70550021	Bracket coupling
23	2	70550020	Cover for TA1
24	1	70550013	Uni-lap Control
25	1	70550043	Sticker ELD
26	4	70519931	Leg support clamp
27	1	70550012	Charger incl. plug
28	2	70550006	Actuator TA1
29	1	70550018	Chassispart charger
30	2	70560004	Battery Set
31	1	70550007	Manual control
32	2	70530015	Vibration damper
33	1	70530030	Emergency stop
34	1	70550014	Emergency lowering button
35	1	70550045	Switch On/Off
36	2	70530029	Pull knob
37	1	70550046	Sticker E-Stop
38	1	70550044	Sticker on/off
39	1	69501104	Bedpan + lid
40	1	70580005	Guide for bedpan set
41	3	70530025	Castor double roll set
42	4	70530006	Protective cover wheel

Drawn by: ES	Date: 10-10-2013
Revised: 6 /KA	Date: 18-01-2023
Revised: 7 /KA	Date: 29-09-2023

## Overview parts Elexo XXL

Copyright reserved  
by law

Unit of measure-  
ment: mm

Lopital Nederland bv

Type no: 51005300

## Elexo XXL 51005300

Part nr.	QTY.	Art.nr.	Part
43	8	70602006	Snaploc
44	4		Tube cap Ø28
45	4		Flanged Slide bearing 16x20x17
46	2		Flanged Slide bearing 20x23x16.5
47	4		Flanged Slide bearing 10x12x15
48	4		Washer PA M20
49	2		Countersunk screw M6x25
50	4		Tube clamp Ø16
51	4		Cylinder head screw M6x35
52	10		Washer M10
53	2		Locknut M10
54	4		Tree clip
55	4		Capring M4
56	4		Chipboard screw 4,5x25
57	4		Screw cap M4
58	12		Retaining ring 12
59	2		Chipboard screw 3,9x16
60	6		Cylinder head screw M10x80
61	4		Cylinder head screw M14x55 Dacromet
62	2		Cylinder head screw M5x10
63	2		Snap nut M4
64	4		Cylinder head screw M8x35
65	4		Locknut M8
66	4		Nord-Lock M10
67	8		Washer M6 A4
68	8		PAN head cross slotted screw M6x16
69	4		Washer M5 Ø15
70	4		PAN head cross slotted screw M5x8
71	2		Tube cap Ø25
72	1	70530028	Battery cable with fuse



Drawn by: ES	Date: 10-10-2013
Revised: 6 /KA	Date: 18-01-2023
Revised: 7 /KA	Date: 29-09-2023

## Overview parts Elexo XXL

Copyright reserved  
by law

Unit of measure-  
ment: mm

Lopital Nederland bv

Type no: 51005300