



Instruction for use
Tango XL
shower toilet chair



Instruction for use

Tango XL

5100 5750

shower toilet chair

Basic UDI-DI: 872025610370751005750MU



 **lopital**[®]
If caregivers have a choice

Important:

Read these instructions carefully before using the Tango XL!

Date first used.....

User.....

Name.....

Address.....

Manufacturer

Lopital Nederland B.V.

Laarakkerweg 9

















5061 JR Oosterwijk

T +31 (0)13 5239300

www.lopital.nl

info@lopital.nl

All of the aids produced by Lopital comply with the applicable standards for Class I medical products (MDR (EU) 2017/745).

Explanation of symbols in 'instructions for use' and on labels			
	Note		Follow the instructions for use.
	Date of production		Read cleaning and disinfecting protocol.
	Manufacturer information		Antibacterial
	Article number		Temperature
	Serial number		Air pressure
	Conformité Européenne		Humidity level
	Medical device		Maximum load
	GS1 Datamatrix containing the following information: (01) Global Trade Item Number (internationally unique item number) (11) Production date (21) Serial number		Keep the device away from heat, hot surfaces, sparks, open flames, and other ignition sources. No smoking.

Contents

1.	Introduction	5
1.1.	Introduction	5
1.2.	Safety.....	5
1.3.	Warranty.....	7
2.	Use	7
2.1.	First use.....	7
2.2.	Working with the Tango XL	9
3.	Operation.....	9
3.1.	Wheel blocking.....	9
3.2.	Directional / braked wheel	10
3.3.	Footrests.....	10
3.4.	Armrests.....	12
3.5.	Toilet function.....	12
3.5.1.	In combination with toilet	12
3.5.2.	Use of a bedpan or bucket	12
3.6.	Removable seat.....	13
4.	Cleaning and maintenance	14
5.	Environmental conditions	15
6.	Removal of parts.....	15
7.	Technical specifications.....	16
7.1.	Structure	16
7.2.	Dimensions	16

1. Introduction

1.1. Introduction

The Tango XL shower-toilet chair is a medical device for caring and washing clients in an ergonomically responsible manner. Comfort is the primary consideration for both the client and the carer. The Tango XL can also be used as a toilet chair, in combination with a bedpan / bucket. The Tango XL is a shower-toilet chair with independent, folding armrests, polyurethane two-piece removable seat, rails for bedpan and rotating and foldable footrests.



The frame of the Tango XL has an antibacterial coating. The coating contains silver ions. Antibacterial coating based on silver ions prevents dangerous micro-organisms such as bacteria, fungi and algae from multiplying and spreading. The effectiveness was confirmed in several tests. Germs were demonstrably reduced in a short time to 99.99%. The coating only affects bacteria that are found on the surface and not on airborne micro-organisms. The coating does not protect users or others from disease-causing bacteria, germs, viruses or other harmful organisms. On a dirty surface, the coating loses its antibacterial properties so proper cleaning and disinfection is still a must.

The average technical lifecycle of the Tango XL is 10 years, assuming it is cleaned daily and regularly maintained according to the maintenance schedule supplied (*See chapter 4. Cleaning and maintenance*).

The Tango XL is a Class I medical device with CE marking. This marking is in accordance with the regulation MDR (EU) 2017/745.

1.2. Safety



A failure to comply with the following safety points and the further definitions in these instructions for use could lead to hazardous situations. Please read carefully before use.

Use

- Make sure the Tango XL brake is applied before transferring clients.
- Notify the client before moving the Tango XL.
- Make sure that the client always places his / her arms on the armrests provided to prevent possible entrapment.
- Keep the armrests down / closed when moving the Tango XL.
- Always try to move backwards over obstacles.
- It is not permitted to charge the Tango XL with more than 250 kg / 550 lbs.
- The Tango XL is only suitable for moving and caring for one person.

- Only use original parts, supplied by Lopital.
- Use the Tango XL in a clean and tidy working environment.
- The Tango XL may only be operated by a qualified person.
- Swimming pool locations are not considered to be 'normal conditions' and will substantially shorten the lifespan of certain components.
- The Tango XL cannot be used on a surface with a gradient over 5 degrees laterally.
- The Tango XL cannot be used on a surface with a gradient of over 10 degrees forwards / backwards.
- The Tango XL may only be used indoors.

Maintenance

- The Tango XL must be inspected and serviced at least once a year.
- In the event of damage to the seat, back or backrest, always replace them for reasons of hygiene.
- Constructive components of the Tango XL may only be replaced by suitably qualified Lopital personnel.
- Service and maintenance must not be carried out while a client is sitting on the chair.
- If erroneous use, transport, an accident or improper maintenance leads to deformation of the Tango XL, it may no longer be used and you must contact the supplier.

In the event of breakdowns, you must contact Lopital or your local supplier.

E-mail: info@lopital.nl



The Tango XL shower-toilet chair should only be operated by competent personnel who have experience in working with the user target group and have watched the instruction/training videos on the Lopital website.



The Tango XL has been developed and produced for use in a professional healthcare environment.



Making changes to the construction may affect the safety of the Tango XL. Furthermore, the liability and guarantee conditions of Lopital lapse. As a result, the Tango XL no longer complies with the MDR (EU) 2017/745 regulation for medical devices.



If a serious incident occurs related to this medical device that affects the caregiver or the client, it must be reported to Lopital or the distributor of the medical device.

1.3. Warranty

All Lopital products are supplied with a 5-year warranty, on construction and / or material faults, unless otherwise stated. In the event of varying warranty terms, the exact period will be indicated on your invoice; this also serves as the warranty itself. Components and batteries that are subject to wear and tear will not fall under warranty unless there are clear construction and / or material faults.

2. Use

2.1. First use

In the picture below, the components of the Tango XL are named. In the event of damage, these parts can be replaced by qualified persons.





<p>Laarakkerweg 9 5061 JR Oisterwijk The Netherlands T: (+31)13-5239300</p> <p>CE</p> <p>Weight: -- kg / -- lbs</p>	<p>GTIN (01) 8720256103---</p>	<p>Item no: -----</p>
	<p> (11) YYMMDD</p>	
	<p>SN (21) -----</p>	
	<p> -- kg / -- LBS</p> <p>Product type</p>	



<p></p> <p>Scan for information</p>

2.2. Working with the Tango XL

Step 1

If necessary, fold the armrests up (See paragraph 3.4. Armrests).

Step 2

Turn the footrests out of position (See paragraph 3.3. Footrests).

Step 3

The Tango XL must be put on the brakes before transferring persons (See paragraph 3.1. Wheel blocking).

Step 4

The client can take a seat in the chair. Support the client if necessary.

Step 5

Turn the footrests back in position and place both feet on the footrests (See paragraph 3.3. Footrests).

Step 6

Fold the armrests again once the client has taken a seat in the chair (See paragraph 3.4. Armrests).

Step 7

Notify the client about the processes you will be carrying out.

Step 8

Release the brakes of the wheels when you wish to move the chair (See paragraph 3.1. Wheel blocking).

3. Operation

3.1. Wheel blocking

- The brake can be applied on the Tango XL by pushing the blue lip (1) on the wheel downwards (See figure 1).
- In order to take the Tango XL off the brake, the same lip on the wheel must be moved upwards by tapping the upper side of the lip (2) (See figure 1).

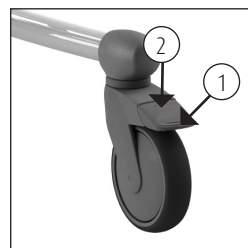


Figure 1. Swivel wheel.

3.2. Directional / braked wheel

The Tango XL has a standard design with a directional / braked caster. This makes pushing along greater distances easier because the seat will drift less sideways. The directional / braked caster is fastened in front.

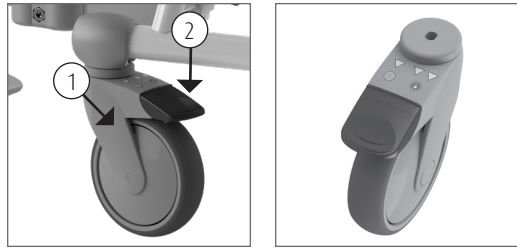


Figure 2. Directional / braked wheel.

To block the wheel direction, push down the black lip. N.B. Make sure the black lip moves 1 position downwards (1) (See figure 2) to lock the direction. When the black lip moves 2 positions downwards, the wheel will be blocked. In order to lock the direction again, the same lip on the wheel must be moved upwards by tapping on the upper side of the black lip (2) (See figure 2).

3.3. Footrests

- Before the client takes a seat in the Tango XL, the footrests must be raised slightly and turned outwards. There is now sufficient room in front of the chair and the client can easily take a seat.
- Once the person has taken a seat, the carer or client raises one leg, the carer then turns the footrest forwards until it falls into the lock. The client can then rest their foot on the support.
- The same procedure is then carried out with the other leg.
- The footrests of the Tango XL are adjustable in length. To do this, pull off the tube adjuster and select the desired position. Then push the tube adjuster back onto the frame.

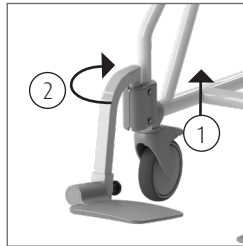


Figure 3. Turning footrest away.



Figure 4. Footrests.



Always put this component back on the chair when you have finished using it. This will prevent the component from being lost.

Option: Comfort leg supports.

The comfort leg supports provide the legs with additional support. The comfort leg supports can be adjusted in length as well as according to the sitting angle. The angle of the leg supports can be adjusted to 4 positions (See figure 5 to 8). To change the seating angle, you have to release the

locking pin. When you have determined the correct position, lock the sitting angle of the comfort support once again (See figure 9).



Make sure that the locking pin slips well into one of the 4 holes.

To adjust the length of the leg supports, you have to release the locking pin underneath by loosening it a bit (See figure 9). Choose the right length for the leg support and tighten the locking pin again.

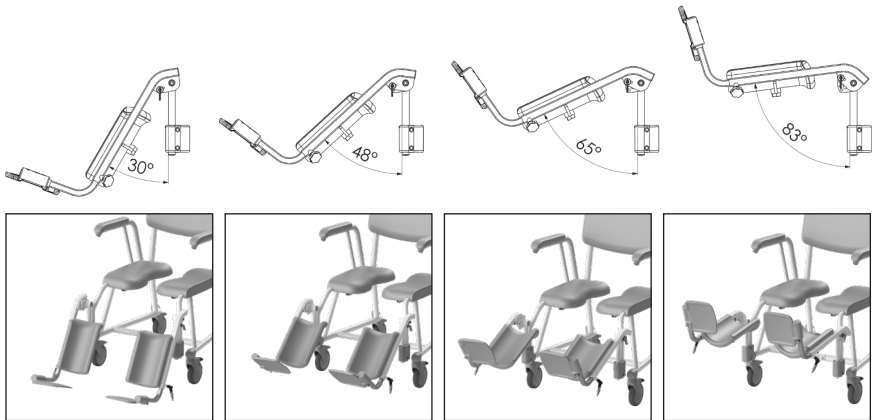


Figure 5 to 8. Adjustable seating angle.

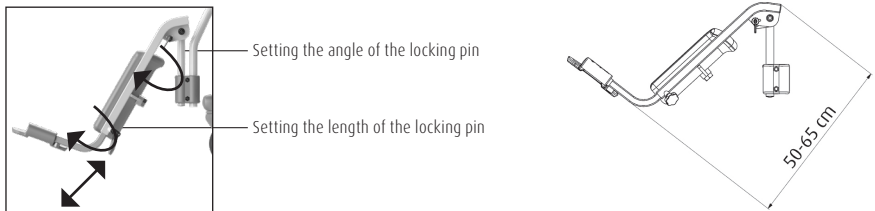


Figure 9. Locking pins.



Make sure that both the sitting angle and the length of the leg support are locked properly before they have to support a load.



Always place these parts on the seat as soon as you finish using them. This prevents the parts from getting lost.

3.4. Armrests

- The independent folding armrests are easy to fold up.
- The blue button ensures the armrest is locked in the lowered position. The caregiver must press the blue button to move the armrest upward.
- Protect the user when moving the armrests upwards and downwards.
- In the folded-up position, they should not fall back down by themselves; in other cases: consult your supplier.



When folding the armrests away, avoid the hinging point to prevent possible entrapment.



When folding the armrests away, avoid the fingers or body parts becoming entrapped between the armrest and the back support.



The armrests or front closure must not be used as a lever arm (e.g. when moving the chair over an obstacle) or as a push / pull bar. You must not sit on it either. This could damage the Tango XL and / or present a hazard.

3.5. Toilet function

3.5.1. In combination with toilet

- The Tango XL can be moved backwards over the toilet.



The Tango XL does not fit over every toilet.

3.5.2. Use of a bedpan or bucket

- If the Tango XL is being used as a toilet chair, the special guides under the seat can be used to accommodate a stainless steel / plastic bedpan or bucket.
- The bedpan or bucket is slid in from the back of the chair (See figure 12).
- If you are using the Lopital plastic bedpan, you must slide it as far forwards as possible (See figure 13).
- If you are using a stainless steel bedpan or bucket, the bedpan can be placed in 2 positions under the seat. At the front of the seat



Figure 10. Armrest.



Figure 11. Armrest folded away.

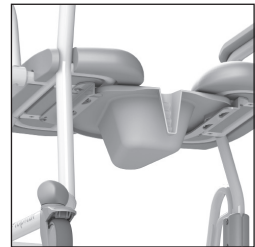


Figure 12. Bedpan.

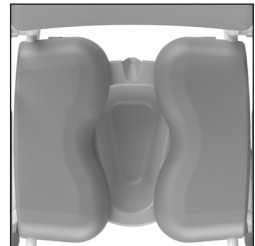


Figure 13. Lopital plastic bedpan.

(See figure 14) or at the back of the seat (See figure 15).



In order to avoid injuries, we advise the carer to insert the bucket or bedpan before the client takes a seat on the chair.



The Lopital plastic bedpan, stainless steel bedpan and bucket must be cleaned and disinfected after use, for example in a bedpan washing unit.



If the Lopital plastic bedpan, stainless steel bedpan or bucket has been used, it must be removed from the chair before the chair is moved or adjusted in height.

3.6. Removable seat

If required, the seat of the Tango XL can be taken off the frame. By doing so, the seat can be cleaned even better. The seat can be taken off the frame without using tools. Pull the seat upwards in the front (See figure 16). To put the seat back, press the seat back onto the frame. Ensure that the seat fits well around the frame.

4. Cleaning and maintenance

The Tango XL must be cleaned after every use, with water and regular household cleaning agents. PUR integrated foam components with IMC coating should be rubbed clean and disinfected. Do not use any abrasive or other aggressive cleaning products. If necessary, the PUR components can be disinfected with a solution of 70% ethanol or propanol.

The time the products must be left on the components has a significant impact upon the lifespan of the PUR components. As a result of the material properties of PUR, contact with sharp objects must be avoided. This could lead to damage or tears in the material. Finally, we would like to point out that non-critical parts of the PUR components must also be cleaned and disinfected.

Check the wheels regularly for hairs, dust and soap residues and remove when necessary. Maintenance and repairs to the Tango XL may only be carried out by qualified personnel.

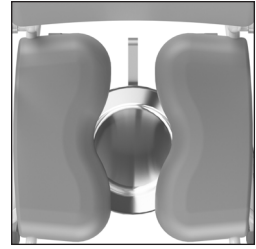


Figure 14. Stainless steel bedpan, front.

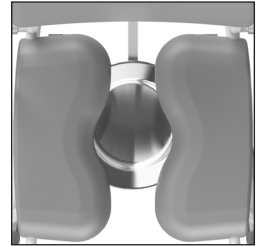


Figure 15. Stainless steel bedpan, back.




Figure 16. Seat.

Maintenance schedule	
After every use	<ul style="list-style-type: none"> • Clean chair with water and regular household cleaning products. • PUR foam components must be cleaned and disinfected. • Check to ensure all parts of the Tango XL are present, including the footrests.
Weekly	<ul style="list-style-type: none"> • Check wheels for hairs, dust and soap residues and remove if necessary. • Check that the chair is still working correctly.
Annually	<ul style="list-style-type: none"> • According to the regulation MDR (EU) 2017/745, medical devices, a chair must be maintained according to the manufacturer's guidelines. This process will preferably be carried out by Lopital or another qualified body. Lopital offers the option of having the chair thoroughly inspected and serviced once a year via Periodic Preventative Maintenance. <p>For more information about the maintenance contract, please contact Lopital (info@lopital.nl) or speak to your supplier.</p>


For a detailed explanation regarding cleaning and disinfecting, see the document 'Cleaning and Disinfection Protocol' on our website www.lopital.com/downloads.

5. Environmental conditions

Conditions during storage and transport

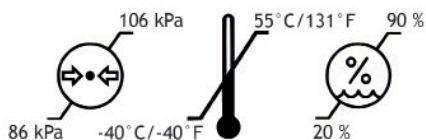

Conditions during use

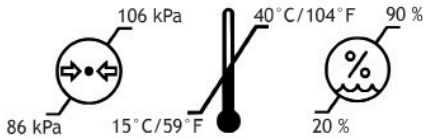

6. Removal of parts

All parts that are replaced or removed can be sent back to Lopital BV. You may also offer the parts to the engineer for destruction. We will ensure that all parts are processed in an environmentally-friendly manner.


7. Technical specifications

7.1. Structure

Frame:	Stainless steel and steel tubes and plates that are equipped with a two-layer powder coating
Hinge points:	Stainless steel, cover caps made of plastic V0
Wheels:	Plastic, Ø 12.5 cm / 4.9 inch, double blocking facility
Body support:	PUR (polyurethane).
Weight:	27 kg / 59.2 lbs
Maximum permitted load:	250 kg / 550 lbs
Weight comfort leg support:	3.6 kg / 8 lbs per unit

7.2. Dimensions

Surface area:	67 cm / 27.2 inch wide, 109 cm / 42.9 inch deep
Seat height:	53 cm / 20.9 inch

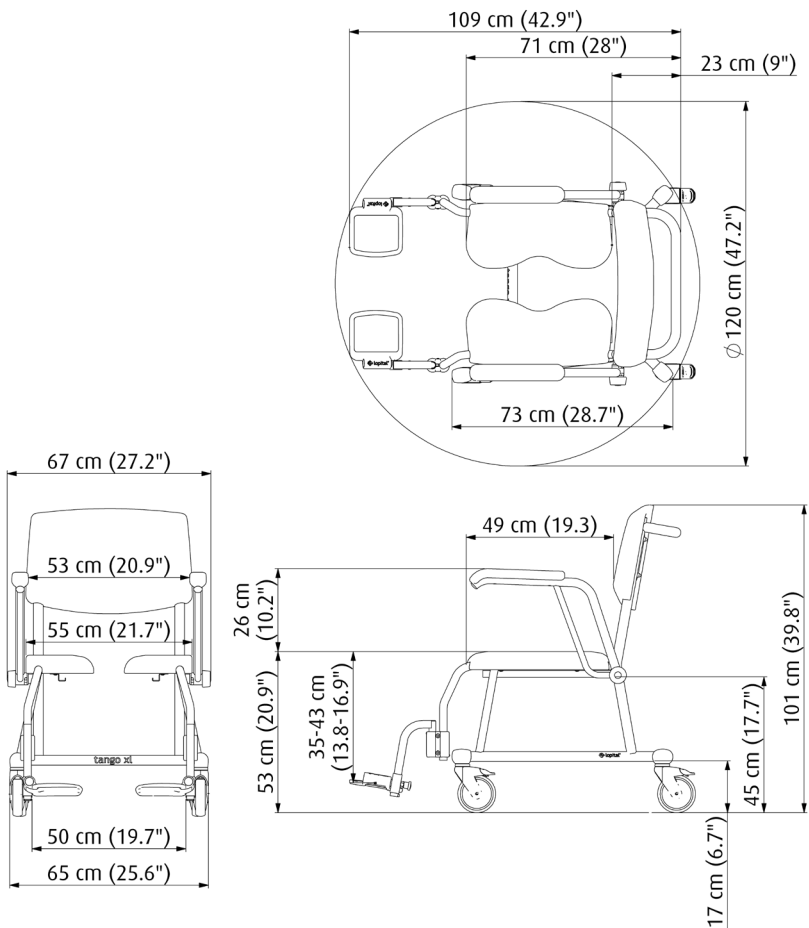



For more information also visit our website: www.lopital.com

Lopital Nederland B.V. Laarakkerweg 9, 5061 JR Oisterwijk, Postbus 56,
5060 AB Oisterwijk, Tel +31 (0)13 52.39.300, E-mail info@lopital.nl,
www.lopital.nl

Copyright ©2025 Lopital. All rights reserved.



Green Deal

 **MADE IN
THE NETHERLANDS**

 **ISO CERTIFIED**
9001 - 13485 - 14001



Scan for
information