

Amfora & Stylex shower trolley & Vienna dressing table

Instruction card

01. Safety recommendations

- Read the instructions for use (www.lopital.com).
- Maximum load 150 kg / 331 lbs (Amfora/Vienna) & 135 kg / 297.6 lbs (Stylex).
- Make sure that the Amfora/Stylex/Vienna is on the brake during client transfers from and to the shower trolley / dressing table



02. Operation (Amfora/Vienna)



6200 4015

Amfora



6500 4025

Vienna




6100 3070

Stylex




03. Operation (Stylex)

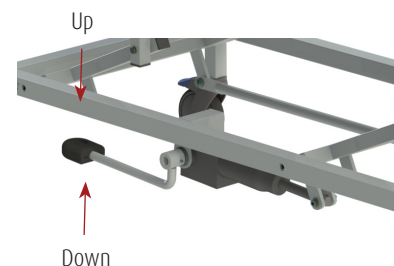


06. Side rail

1. Unlocking
2. Rotate the side rail to the right position

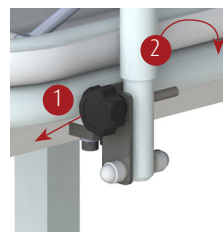

06. Water drainage (Amfora/Stylex)

Water is drained away from the Amfora/Stylex via a drainage hose.

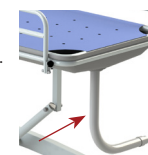

07. Cleaning and maintenance

The Amfora/Stylex/Vienna must be cleaned after every use, with water and regular household cleaning agents. Do not use any abrasive or other aggressive cleaning products. If necessary, the mattress can be disinfected with a solution of 70% ethanol or propanol. The complete maintenance schedule and cleaning protocol can be found in the instruction for use (www.lopital.com).

According to the regulation MDR (EU) 2017/745 for medical devices, the Amfora/Stylex/Vienna must be maintained in line with the manufacturer's instructions. Those instructions can be found in the maintenance protocol, see website Lopital (www.lopital.com).

Important!

This quick instruction card may not be considered a replacement for the instruction for use.

In the event of a breakdown contact Lopital.

Tel: +31 (0) 13-5239300

E-mail: info@lopital.nl

Instruction for use
For the full instructions for use go to: www.lopital.com



ISO CERTIFIED
9001 - 13485 - 14001

INS4015-2026-01 EN