



Safety recommendations


- Read the instructions for use. (www.lopital.com).
- Maximum load 150 kg / 331 lbs (Amfora/Vienna) & 135 kg / 298 lbs (Stylex)
- Make sure that the Amfora/Stylex/Vienna is on the brake during client transfers from and to the shower trolley / dressing table.




Hydraulic control (Stylex)



Hydraulic control (Stylex)

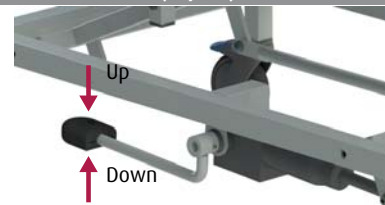


Wheelblocking

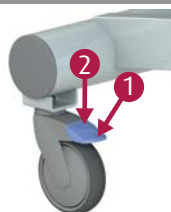

1. Wheel blocking
2. Unlocking

Side rail

1. Unlocking
2. Rotate the side rail to the right position.

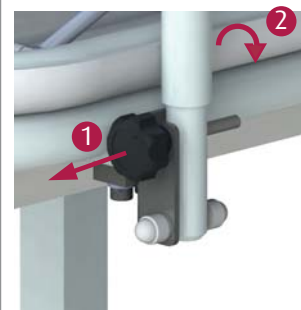

Water drainage (Amfora/Stylex)

Water is drained away from the Amfora/Stylex via a drainage hose.



Cleaning and maintenance

The Amfora/Stylex/Vienna must be cleaned after every use, with water and regular household cleaning agents. Do not use any abrasive or other aggressive cleaning products. If necessary, the mattress can be disinfected with a solution of 70% ethanol or propanol. The complete maintenance schedule and cleaning protocol can be found in the instructions for use. (www.lopital.com)

According to the regulation for medical devices, a Amfora/Stylex/Vienna must be maintained in line with the manufacturer's instructions. Those instructions can be found in the maintenance protocol, see website Lopital (www.lopital.com).

In the event of a breakdown, contact your Lopital distributor.

Important!

This quick user manual may not be considered a replacement for the instructions for use.

